



Quick Guide - Combi-lock



1

Position the traverse rail so it is aligned with the Combi-lock rail.



2

When the rails are aligned the safety gates open automatically.



3

After checking that the Safety lock has engaged, position yourself right behind or in front of the hoist. It is now safe to manoeuvre the hoist through the Combi-lock, and on to the fixed or traverse rail.



4

When the hoist has passed through the Combi-lock it is safe to move the traverse rail.

WARNING



*Do **NOT** move the traverse rail when the hoist is in this position!*



*GH1Q should **NEVER** be used in systems with a Combi-lock, as there is a risk of the hoist falling down.*



Quick Guide - Combi-lock



Place the hoist at the yellow marking approx. 1 meter from the end of the traverse rail and Combi-lock.



Place the user facing / looking lengthwise the traverse rail and in a level where you can push / hold as in the picture.

To align the rails and engage the Combi-lock place yourself on the side of the user so you push or pull perpendicular on the traverse rail.

If the Combi-lock doesn't engage the first time, move the traverse rail approx. $\frac{1}{2}$ to 1 meter away from the Combi-lock rail and try again.



When the Combi-lock has engaged position yourself behind the user and push lengthwise the traverse rail through the Combi-lock. The user is still facing / looking lengthwise the traverse rail.