DK Positionsås 2
GB/US Positioning Lock 6
DE Feststellbremse für Traverse und Hebemodul 11
SE Positionsås 16
NO Posisjonsås 20
FR Verrouillage Position Moteur 24
IT Blocco di posizionamento 29
NL Positioning Lock 33

User manual – vers. 102.0

MD CE

Guldmann™

DK... POSITIONSLÅS

Vers. 102.0

Varenr.:

Positionslås	557600
Positionslås - Single	557650

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Vers. 102.0

Item nos:

Positioning Lock	557600
Positioning Lock - Single	557650

1.00 Use of Positioning Lock	6
1.01 Manufacturer	6
1.02 Intended purpose	6
1.03 Area of use	6
1.04 Conditions of use	6
1.05 Important/Precautions	6
1.06 Mounting a new Positioning Lock in an existing hoist/rail system	6
2.00 Description of features	7
2.01 IR remote control	7
2.02 Indicator light	8
3.00 Transport and storage	8
4.00 Maintenance and storage	8
4.01 Cleaning and disinfection	8
4.02 Storage	8
4.03 Daily maintenance	8
4.04 Disposal of Positioning lock, including battery	8
5.00 Service and lifetime	8
5.01 Lifetime	8
5.02 Safety/service inspections	8
5.03 Accessories	8
6.00 Classification	9
7.00 EU-Declaration of conformity	9
8.00 Environmental policy statement - V. Guldmann A/S ..	9
9.00 Warranty and service conditions	9
A. Warranty	9
B. Service or Repair	9

1.00 Use of Positioning Lock

1.01 Manufacturer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. + 45 8741 3100
www.guldmann.com

1.02 Intended purpose

The Intended purpose of Guldmann rail components is to assist moving, routing and locking the hoist and railsystem.

1.03 Area of use

The Positioning Lock is suited for use in hospitals, nursing homes, institutions, rehabilitation centers, riding schools and in private homes.

1.04 Conditions of use

The Positioning Lock is used to lock a hoist in a single rail system or to lock a traverse rail system in a certain position. It is possible to lock a hoist and traverse rail individually or together at the same time.

The Positioning Lock is mechanical and is controlled via an infra-red (IR) remote control. The system/hoist is placed in the desired position and then locked by pressing a button on the IR remote control. To unlock the system, press the remote control once again; *see section 2.00*.

1.05 Important/Precautions

The use of the Positioning Lock is subject to the following prerequisites:

- The Positioning Lock is only designed for installation with the GH3/GH3+ ceiling hoist in A, B or C rails.
- The Positioning Lock may only be used by qualified personnel.
- The hoist may not be used to lift patients when one or more of the locks are activated.
- It is important that the lock is always disconnected before lifting patients (the user)
- The Positioning Lock may not be used with the following Guldmann products: Combi-Lock, Switch Track, Rail Curves Turntable, Drive motor for traverse rail or hoist with drive motor.
- When the positioning lock is activated and the lifting strap is angled more than 45 degrees from the vertical position, the hoist can be loaded with maximum 20 kg.
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority.

1.06 Mounting a new Positioning Lock in an existing hoist/rail system

The Positioning Lock is Class I equipment and must be installed by the Guldmann Service Team or by a certified installer.

Disconnect the equipment from the power supply by flipping the main breaker switch.

The Positioning Lock is equipped with battery that require regular charging. The power supply for charging and the transformer must be connected by the Guldmann Service Team or a certified installer.

The Positioning Lock is charged automatically in one of the parallel rails. This ensures that the Positioning Lock will always work and be ready for use, and that the battery last longer.

The transformer must be connected and switched on before charging can take place. A green indicator light on the transformer indicates that it is connected and turned on. Always use the transformer supplied.

Safety concerning static electricity (ESD)


Service technicians and installers must use an ESD-safety package consisting of a mat, a ground wire, and a bracelet. The technician/installer connects the mat to a grounding point, for instance a radiator or a water pipe. The technician/installer must then put on the bracelet and connect it to the mat. If it is not possible to find a grounding point, the mat and the bracelet must be used as a minimum.

Only then is it allowed to work with the PC Board or components where it is possible to come into contact with the PC Board.

2.00 Description of features

2.01 IR remote control

The Positioning Lock is operated via IR remote control. Press the button once to activate the desired function.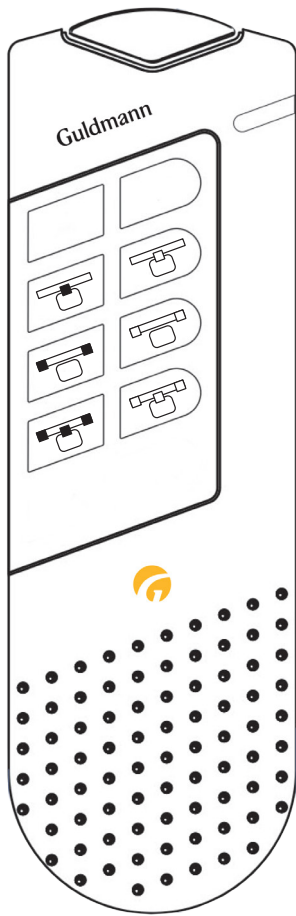


The following functions can be activated by using the IR remote control for the Positioning Lock:


- Lock (red)
- Unlock (green)

Button	Function	Movement
	Lock hoist	Only movement for traverse rail
	Unlock hoist	
	Lock traverse rail	Only movement for traverse rail
	Unlock traverse rail	
	Lock traverse rail and hoist	Only movement for hoist and hoist
	Unlock traverse rail and hoist	


Lock hoist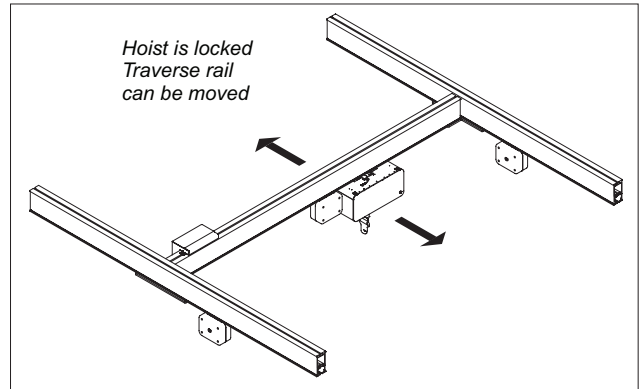

Lock traverse rail

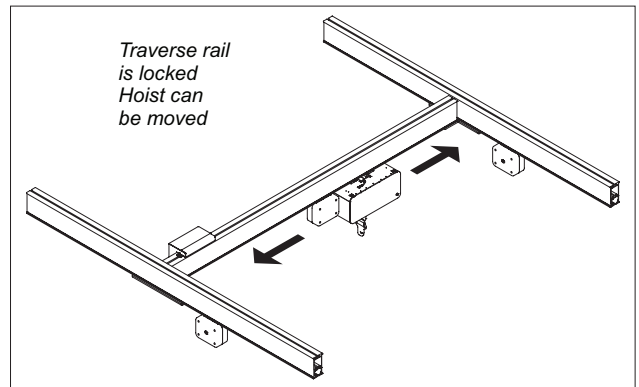Lock hoist and traverse rail

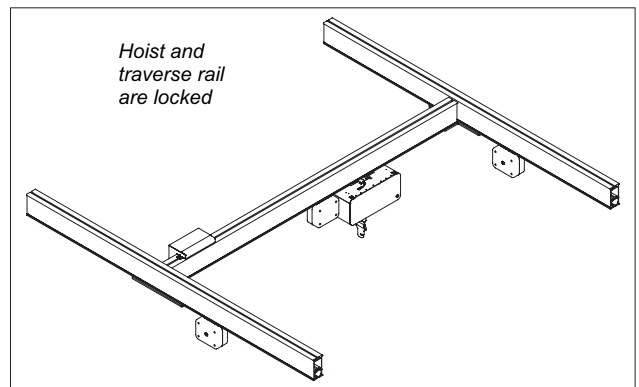2.02 Indicator light


The positioning lock has a visible, indicator light on the underside of all the locking devices.

Indicator light	Status	Comments
Off	Off / stand-by / unlocked	
Constant light	Locked	
Continuous flashing	Activation / de-activation of lock	
3 flashes within 0,5 seconds	Signal reception	
5 x 2 flashes followed by a 3 second pause	Low battery	Sequence starts and is run once when the situation occurs
5 x 4 flashes followed by a 3 second pause	Over-current	Sequence starts and is run once when the situation occurs
Continuous flashing	Lost position	When the positioning lock is connected to the battery at installation or at power outage, the indicator light will flash continuously as if the lock is being activated or de-activated. This is because a situation arises where the lock is unable to see whether it is locked or unlocked when it is activated after a power outage. Press the "unlock traverse rail and hoist" on the remote control to solve this.

3.00 Transport and storage

Guldmann recommends that the products are always transported and stored in the original packaging.

Key to symbols on the packaging:

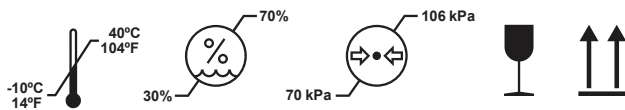The products should be stored at:

- Temperatures between -10 and +40°C / 14 and 104°F
- A relative air humidity of between 30 and 70 %
- An air pressure of between 700 and 1060 hPa
- This side up

4.00 Maintenance and storage

4.01 Cleaning and disinfection

We recommend that the products and the parts patients and caregivers can come in contact with, are cleaned with a damp cloth using warm water and a mild soap solution.

When disinfection is needed, use disinfectant wipes with up to a 85% solution of isopropyl, or a damp cloth using warm water and a disinfectant cleaner, e.g. an chlorine dissolving up to 1500 ppm.

If other chemicals and/or liquids with higher resolution should be used to clean or disinfect these products, please contact Guldmann providing the item's safety sheet chemical composition for consideration.

Caution: Take great care to ensure that no liquids get inside the hoist and Positioning lock. These are not waterproof. Failure to protect the hoist and Positioning lock from liquids may result in damage or may cause personal injury.

4.02 Storage

See 3.00

For long-term storage, disconnect the battery plugs and the plug from the battery at the charging PCB.

4.03 Daily maintenance

Ensure that the Positioning Lock is intact. Do not use the Positioning Lock if it is damaged or defective. Instead, contact the Guldmann Service Team or a technician certified in accordance with Guldmann's indications.

4.04 Disposal of Positioning lock, including battery

Local and national regulations on environmentally correct recycling must be observed. Batteries (type NiMH) must always be delivered to an approved recycling point.

5.00 Service and lifetime

5.01 Lifetime

The products have an expected lifetime of 15 years, on the condition of correct use and correct service inspections, see section 5.02.

Replacement of components

Replacement of battery and PCBs must be performed by a qualified service technician or the Guldmann Service Team.

5.02 Safety/service inspections

In accordance with international standard EN/ISO 10535 "Hoist for the transfer of disabled persons – Requirements and test methods" an inspection should be performed on the hoist at least once a year. Guldmann recommends that regular safety/service inspection is performed at least once a year with regard to the pattern of usage.

Safety/service inspections of the products must be performed by a qualified service technician or the Guldmann Service Team. In connection with the purchase Guldmann may offer a service agreement for this inspection.

During the safety/service inspection a report must be prepared on what was checked and replaced. Parts that are worn or defective must be replaced with new parts from Guldmann. Spare parts drawings and documentation can be obtained from the manufacturer or supplier.

Documentation/checklist regarding safety/service inspection can be obtained from the manufacturer or supplier.

5.03 Accessories

Infrared remote control

Guldmann item number 935039

Batteries

NiMH battery 24V / 2.1Ah, Guldmann item number 550574

Transformer

Transformer, Class I, Guldmann item number 550200

6.00 Classification

CE marking

Read the manual before use

Medical Device Class I
in accordance with EU MDR Regulation


Must not be disposed of as standard household waste,
must be recycled.

Class I equipment: Permanent installation with protective ground


The equipment is not suitable for use in the presence of flammable mixtures.

Examples of serial number label

Positioning Lock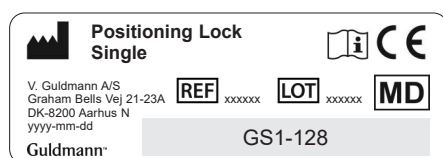

Transformer Class I

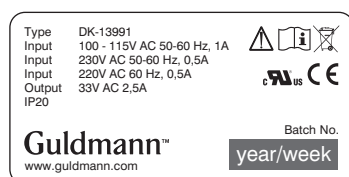7.00 EU-Declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

8.00 Environmental policy statement - V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

9.00 Warranty and service conditions

A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B. Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann Sverige AB
Guldmann Srl
Guldmann Inc.