GB/US	Swing Lift II	2
DE	Swing Lift II	9
SE	Swing Lift II	16
NO	Swing Lift II	23
FR	Swing Lift II	30
IT	Swing Lift II	37
NL	Swing Lift II	44

User manual – vers. 101.00

Guldmann™

GB Swing Lift II

Item no:
17200

1.00	Purpose and use	3
1.01	Manufacturer	3
1.02	Intended Purpose	3
1.03	Area of use	3
1.04	Conditions of use	3
1.05	Important/Precautions	3
1.06	Installation	4
1.07	Accessories	4
2.00	Service and lifetime	4
2.01	Lifetime	4
2.02	Safety/service inspections	4
3.00	Label and marking	5
4.00	Technical specifications	5
5.00	EU-Declaration of conformity	6
6.00	Environmental policy statement - V. Guldmann A/S	6
7.00	Warranty and service conditions	7
A.	Warranty	7
B.	Service or Repair	8

1.00 Purpose and use

1.01 Manufacturer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. +45 8741 3100
www.guldmann.com

1.02 Intended Purpose

Free-standing systems is intended to support ceiling hoist in a movable systems that is not suspended from a building or structure.

1.03 Area of use

The swing lift is suitable for use in swimming pools, boat landings, etc. to move people in and out of boats or pools of different kinds. The lift can also be mounted on larger boats.

1.04 Conditions of use

The swing structure can be moved to another base structure placed in a different location. The weight of the swing structure must be considered. The "extra" base structure must be ordered with the Swing Lift II, because the base and top sleeve is custom made for each Swing Lift II.

1.05 Important/Precautions

- Read the instructions carefully before using the Swing Lift II and in connection with cleaning and service of the hoist.
- The Swing Lift II must be checked physically and visually before use. If damage is suspected, the Swing Lift II must not be used before the distributor has authorized it.
- Check that the rail and wire are fastened properly before use.
- Check that the end stop on the rail is mounted and in place before using the Swing Lift II.
- The Swing Lift II's maximum load must never be exceeded.
- The Swing Lift II may only be used to lift a person.
- If a defect appears during use of the Swing Lift II, stop using the Swing Lift II and contact the Guldmann Service Team for repairs.
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority.

1.06 Installation
Installation of the Swing Lift II must be performed by a qualified service technician or the Guldmann Service Team.

1.07 Accessories
The Swing Lift II is designed to be used with other Guldmann products such as hoist, sling and mastlift. Rain cover is also available.
Obtain a special brochure from your distributor, manufacturer or at www.guldmann.com

Extension strap on the hoist

The extension strap is used where the distance between the lower part of the rail and the floor exceeds 3,5 m (11'7"). The extension strap is available as an accessory.

2.00 Service and lifetime

2.01 Lifetime
The Swing Lift II has an expected lifetime of 15 years, on the condition of correct use and correct service inspections, see section 2.02.

Replacement of components

Replacement of batteries, PCBs and lifting strap on the hoist must be performed by a qualified service technician or the Guldmann Service Team.

2.02 Safety/service inspections
In accordance with international standard EN/ISO 10535 "Hoist for the transfer of disabled persons – Requirements and test methods" an inspection should be performed on the Swing Lift II at least once a year.
Guldmann recommends that regular safety/service inspection is performed at least once a year with regard to the pattern of usage.

Inspection of the Swing Lift II must be performed by a qualified service technician or the Guldmann Service Team.
In connection with the purchase of the Swing Lift II, Guldmann may offer a service agreement for this inspection.

During the safety/service inspection a report must be prepared on what was checked and replaced. Parts that are worn or defective must be replaced with new parts from Guldmann. Spare parts drawings and documentation can be obtained from the manufacturer or supplier.

Documentation/checklist regarding safety/service inspection can be obtained from the manufacturer or supplier.

3.00 Label and marking

CE marking

Medical Device Class I in accordance with EU MDR Regulation

Read the manual before use

Example of label

4.00 Technical specifications

Functions

Lifting capacity, SWL 205 kg (450 lbs)
Operation Manual

Weight and materials


Swing Lift II 107 kg (235 lbs)
Swing structure 42,5 kg (93 lbs)

Safety

The swing function has been equipped with a pin that prevents the Swing Lift II from moving when it is not in use. This pin must always be activated when the Swing Lift II is not in operation.

Dimensions

A	2500 mm
B	615 mm
C	2910 mm
D	2300 mm
E	500 mm

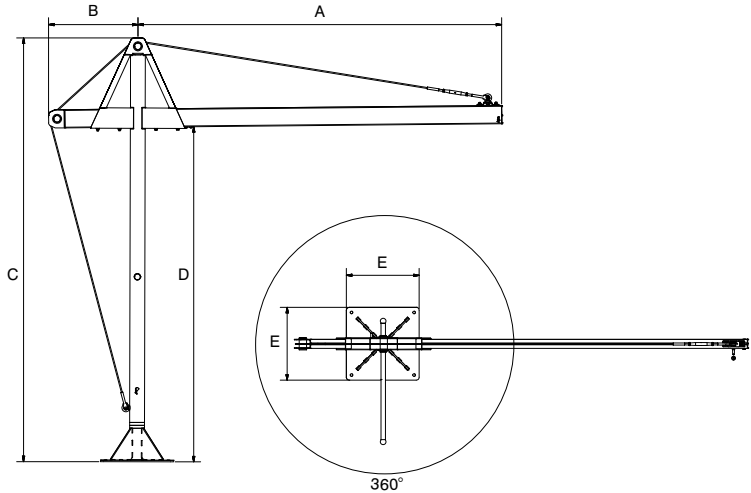5.00 EU-Declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

6.00 Environmental policy statement - V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised.

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

7.00 **Warranty and service conditions**

A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password

are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B. Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl