
DK	GTP1 Transfer platform	3
GB/US	GTP1 Transfer platform	10
DE.....	GTP1 Transferplattform	17
SE.....	GTP1 Transfer platform	24
FR.....	GTP1 Plateforme de transfert	31
IT	GTP1 Piattaforma per il trasferimento	38
NO	GTP1 Transferplattform	45
NL.....	GTP1 Verplaatsingsplatform	52

User manual – vers. 103.0

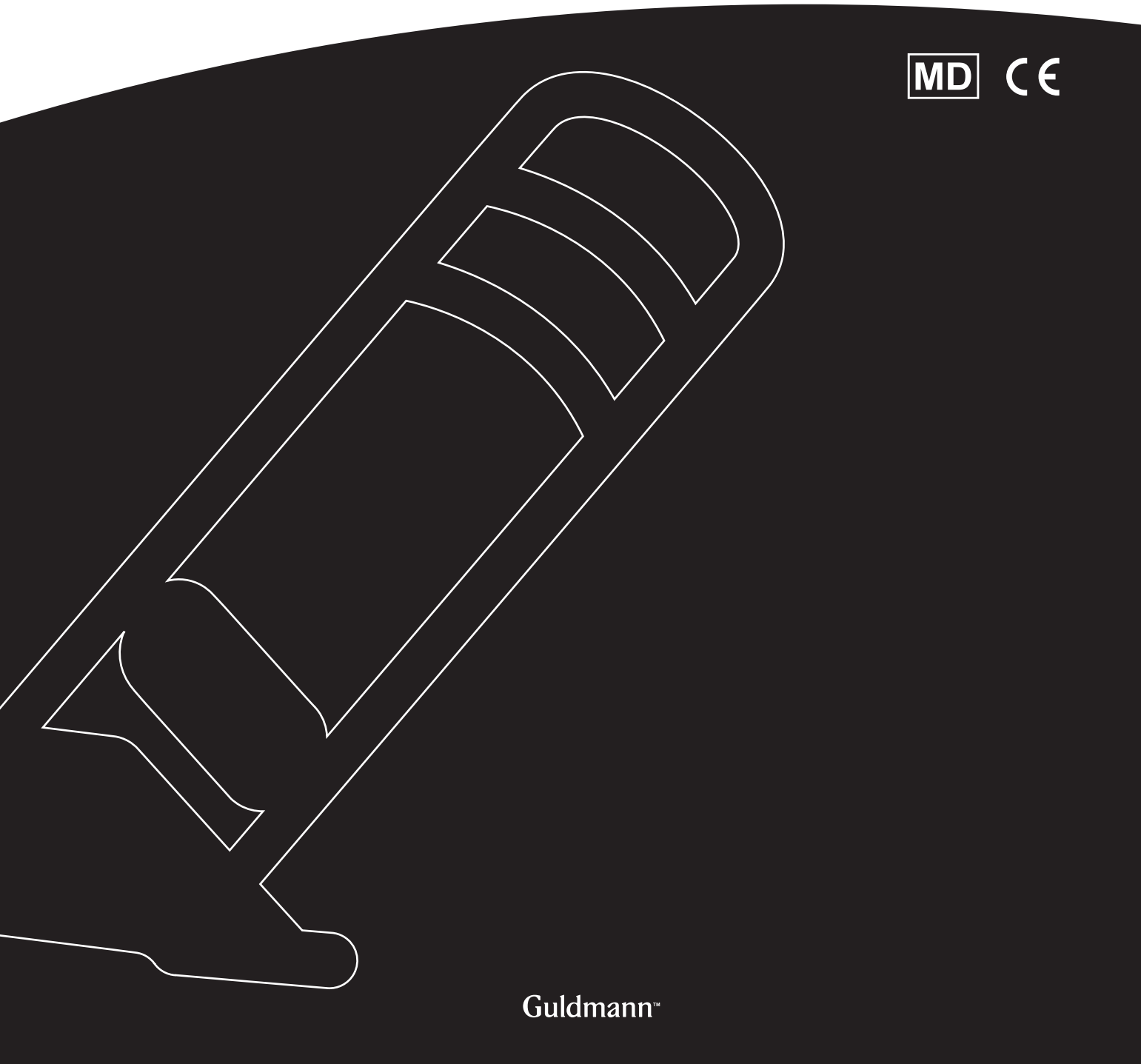Guldmann™

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Vers. 103.0

Item nos:
560700

1.00	Purpose and use	10
1.01	Manufacturer	10
1.02	Intended purpose	10
1.03	Area of use	10
1.04	Conditions of use	10
1.05	Important/precautions	10
2.00	Definitions	11
2.01	Accessories	11
3.00	Unpacking and preparation	11
3.01	Assembly instruction	11
4.00	Checklist before use	11
4.01	Use	11
4.02	Safety functions	14
5.00	Cleaning and disinfection	14
6.00	Service and lifetime	14
6.01	Service check – All-round	14
6.02	Owners daily maintenance	14
6.03	Troubleshooting	14
7.00	Environmental conditions	14
8.00	Technical specifications	15
9.00	Labelling	15
10.00	EU-Declaration of conformity	16
11.00	Environmental policy statement – V. Guldmann A/S	16
12.00	Warranty and service conditions	16
A.	Warranty	16
B.	Service or Repair	16

1.00 Purpose and use**1.01 Manufacturer**

V. Guldmann A/S
 Graham Bells Vej 21-23A
 DK-8200 Aarhus N
 Tel. + 45 8741 3100
 www.guldmann.com

1.02 Intended purpose

GTP1 Transfer platform is intended for transferring a person with disabilities.

1.03 Area of use

GTP1 Transfer platform is suitable for professional use in hospitals, nursing homes, rehabilitation centers, institutions and in private homes and buildings.

- The transfer platform is designed to be used indoors whenever the user needs to be transferred on a level surface
- The transfer platform can be used in wet areas (bathroom only)
- The transfer platform is designed to be used by multiple users

1.04 Conditions of use

GTP1 is a transfer platform designed to assist the user from a sitting to a standing position. The transfer platform is suitable for moving a person who is capable of actively participating in the process. The user must also have good communication skills.

When the transfer platform is used the assumption is that:

- The user can balance in a sitting and standing position, has strength in the lower and upper body and can communicate
- The transfer platform will be operated by a qualified person
- The transfer platform will be used for transfer over short distance only. E.g. bed to chair, chair to toilet, etc.
- The transfer platform is moved at a maximum speed corresponding to normal walking speed
- The transfer platform is used with Guldmann Transfer Belt only
- When moving the transfer platform to another user / ward / room, perform normal hygienic disinfection


Guldmann cannot be held responsible for any faults or accidents that may occur due to incorrect positioning of the belt, nor for inadequate attention paid by the helper. We strongly recommend assessment of the user before every transfer.

1.05 Important/precautions

- Read the user manual carefully before using the transfer platform and in connection with cleaning and service of the transfer platform
- Use the transfer platform to transfer a person only
- Always plan your transfer before commencing
- Use belts that are designed to be used with the transfer platform only. Suitable belts are labelled with "only for GTP1"
- Do not exceed max. rated load
- Use the transfer platform on an even and level surface only
- Do not run the transfer platform into persons or objects
- In case of damage, do not use the transfer platform until authorized by qualified technician
- Do not modify this equipment without the authorization from the manufacturer
- The transfer platform is not intended to drive over steps and thresholds
- The transfer platform should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the transfer platform should be observed to verify normal operation in the configuration in which it will be used
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority

2.00 Definitions

1. Handle
2. Tube, left
3. Tube, right
4. Attachment knob
5. Leg support
6. Release button
7. Center wheel
8. Swivel wheel
9. Baseplate
10. Brake pedal
11. Ideal foot area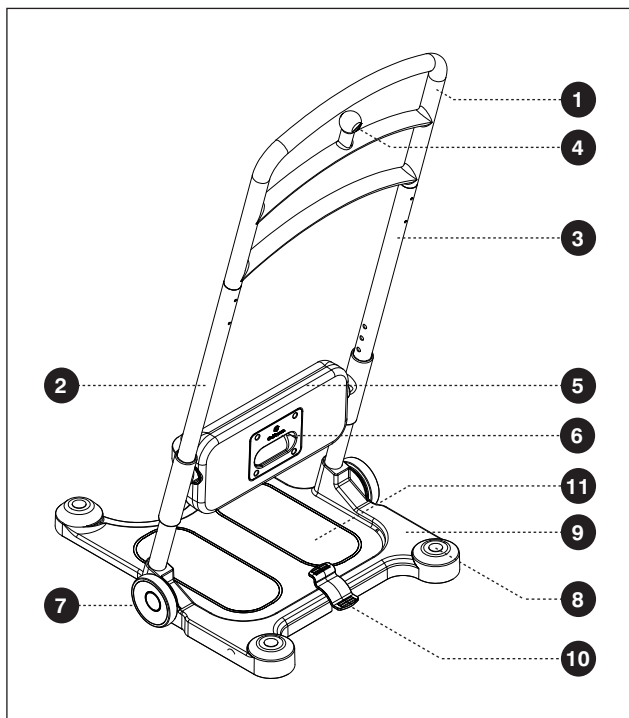

2.01 Accessories

The following accessories are available:

- Transfer Belt, size S
- Transfer Belt, size M
- Transfer Belt, size L

3.00 Unpacking and preparation

Visual check of the transfer platform


If the packaging is damaged on receipt, each part of the transfer platform must be carefully examined for visible defects or deficiencies. In case of suspected damage, do not use the transfer platform until authorized by qualified technician.

3.01 Assembly instruction

Guldmann recommends that the person making the assembly of the transfer platform has a basis mechanical understanding. Unpack the transfer platform on a level surface and gently remove cardboard and protective foam.

Insert the two tubes (pos. 3, 4) into the openings of the baseplate (pos. 2). Make sure that the leg support front (pos. 5) is facing away from the brake pedal (pos. 6). In the baseplate openings, 2 screws (pos. 7) are pre-mounted with retaining washers (pos. 8). Tighten these screws into the thread at the end of the tubes using an Allen key. Make sure that the tubes are mounted correctly, tighten properly and are not wobbly.

To dismantle, untighten screws until the tubes are loose. Make sure that the screws stays mounted into the tubes.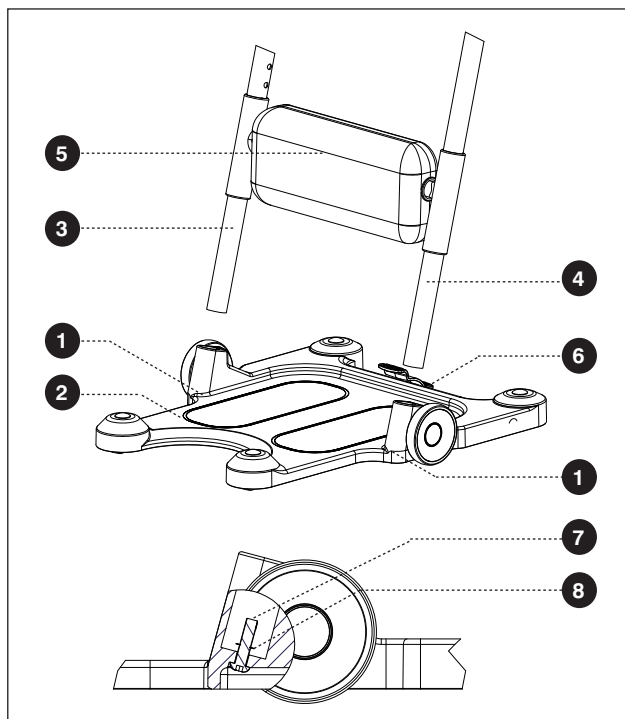

4.00 Checklist before use

- Make sure that the transfer platform is moving freely
- Check the belt for damage or fraying
- Check that the leg support is able to adjust
- Check that the brakes are able to lock the wheels
- If the user is in a wheelchair, commode chair or bed the brakes must be applied

4.01 Use

The transfer platform is designed to transfer from a sitting position only. Before using the transfer platform, we strongly recommend that a full risk assessment is carried out by a qualified person to ensure the user capability, the correct belts are used and the environment is suitable. Anyone using the transfer platform should be fully trained in its use.

Before every transfer with the GTP1, we strongly recommend you carry out a quick assessment of the user's capability.

Working with the transfer platform

Always maintain a good working posture when applying the belt or working with the transfer platform. When moving a person use handle and walk with the transfer platform keeping the load as close as possible to your body. Never pull or twist at arm's length, this can cause injury to the helper.


Also, follow the guidance labels on the tubes.

Quick Assessment

With the user in a seated position in chair / wheelchair / edge of bed – ask the user to push down through his/her knees (feet must be placed flat on the floor) – upon inspection the helper should see the user's quads tighten. This indicates strength to assist with the standing procedure, it also confirms communication skills.

For testing the user's strength in the arms - ask the user to reach their hands to you and ask to clench your fists/hands. The helper should feel the user tighten the hands.

1.

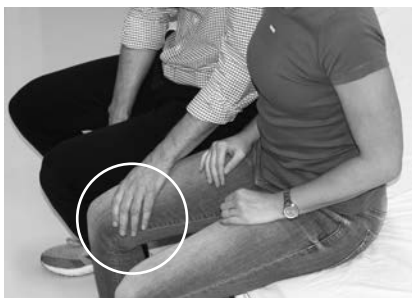Before using the GTP1 Transfer platform ensure the user's functional capability, the user's capability to cooperate and follow an instruction and that the environment is suitable.

- a. **Test of tension of the leg muscles:**
When the user is in a seated position ask the user to push down through the knees – upon inspection and feeling via the hand - the helper should perceive the user's quads tighten.
- b. **Test of tension of the arm muscles:**
Ask the user to clench your fists / hands – the helper should feel the user tighten the hands.

These tests indicate strength to assist with the standing procedure and it also confirms communication skills.

2.

- a. If it is assessed that the belt is needed – place the belt around the low back area when the user is in seated position. Fasten the belt with the elastic belt.

Recommendation: Use the belt in standing position.

- b. Place the user's feet onto the markings on the foot plate.

3.

- a. Adjust the knee pad – the top of the knee pad should be level with a distance of two fingers below the knee joint.

Make sure that the knee pad is locked.

- b. Lock the Transfer platform by stepping on the lower part of the brake pedal.

4.

Encourage the user to lean forward and grab hold of the handle suitable for the user.

Encourage the user to stand up. At the same time the helper stands in front of the Transfer platform, grabs hold of the handle and places one foot on the edge of the foot plate.

5.

- a. Attach the belt to the transfer platform
- b. Check both straps of the belt are mounted correctly at the handle.

Warning: Do not leave the user alone while they are on the transfer platform.

6.

- a. Unlock the brakes by pressing the top part of the brake pedal.
- b. Turn the user to the chair/commode chair/bed etc.

7.

Before the user sits down – lock the transfer platform again – release the belt from the transfer platform and let the user sit down. Remove the belt from the user.

4.02 Safety functions

Use of brakes

Apply the brake by stepping on the lower part of pedal, marked with red. Release the brake, by stepping the top part of pedal, marked with green. Both center wheels are locked when stepping on the lower part of pedal.

Adjustment of knee pad

The knee pad can be adjusted in vertical direction to fit the different users. Make the knee pad adjustable by pushing and holding the button on the backside of the knee pad. To lock the knee pad, release the button and adjust until you hear a click.

5.00 Cleaning and disinfection

For cleaning purposes this product can be used with automatic washing systems without resulting in negative impact on functionality. It is recommended not to exceed a 2 minutes wash with 70 degrees water.

When disinfection is needed, use disinfectant wipes with up to a 85% solution of isopropyl, or a damp cloth using warm water and a disinfectant cleaner, e.g. an chlorine dissolving up to 1500 ppm.

If other chemicals and/or liquids with higher resolution should be used to clean or disinfect these products, please contact Guldmann providing the item's safety sheet chemical composition for consideration.

Caution: Be careful to drain any remaining liquids in the transfer platform after washing. The liquid can damage the equipment and lead to personal injury.

6.00 Service and lifetime

The transfer platform has an expected life span of 10 years. The lifespan is estimated on the basis of correct use, cleaning and maintenance, plus annual service and maintenance carried out by qualified technician. At the end of the expected lifespan, the transfer platform must then be assessed by qualified service staff as to the transfer platforms future use.

No part of the equipment shall be serviced when in use.

Worn or defective parts shall be replaced by new spare parts from V. Guldmann A/S.

Spare parts lists and drawings are available from manufacturer or supplier.

6.01 Service check – All-round

Guldmann recommends that the transfer platform is service checked yearly, according to:

1. Visual check of the product

Check the product for wear and tear
Check the product for any deformity
Check that the product does not show any other deficiencies

2. Test of the product, as during normal use

Check all functions on the product, with and without load (e.g. Knee pad up/down, brake lock/unlock, wheels are moving freely)

3. Check the mechanical condition of the product

Clean the product for dirt and other impurities
Inspect and evaluate the vital parts of the product
Exchange defective and worn parts of the product
Check and tighten all moving parts

4. Point 2 is carried out one more time as a control function

5. Have new errors or problems arisen in point 4?

If new problems have arisen, go back to point 2
If no problems have arisen, finish the inspection

6.02 Owners daily maintenance

The owners' daily maintenance consists of general cleaning. Remove hair and the like from wheels. Make sure that they are able to rotate freely. Test that the brake functions properly.

6.03 Troubleshooting

Error:

The transfer platform is not able to move

A. Do the wheels rotate and move freely?

Yes: See point B

No: Clean the wheels and remove hair and the like or replace with new spare parts

B. Is the brake activated?

Yes: Release the brake

No: See point A

Contact the Guldmann Service Team if the fault cannot be found and corrected.

7.00 Environmental conditions

Operation

The products operational environment:


- Operation temperatures between +10°C and +40°C (50°F and 104°F)
- A relative air humidity of between 30% and 70%
- An air pressure of between 700 hPa and 1060 hPa

Information is illustrated by symbols on packaging including:

- Fragile
- This side up

Beside temperature, the same environmental conditions apply for transportation and storage.

Symbols on the packaging: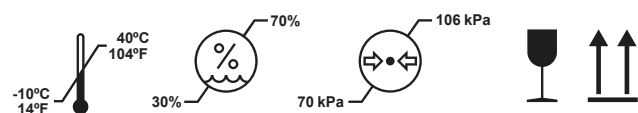

Transportation and storage

The transfer platform can be dismantled for transport and storage.

How to pack the transfer platform for transport

Guldmann recommends that the dismantled transfer platform is always transported in its original packaging. See Assembly instructions.

Storage of transfer platform


The transfer platform should be stored in a dry room, where the humidity does not exceed 70%. The transfer platform must never be stored in bathrooms and similar areas.

Always activate brake function when the transfer platform is put in storage.

How to prevent/avoid corrosion

The transfer platform should not be stored/remain in damp surroundings for long periods of time. Water vapor might liquefy into water on transfer platform, thus causing corrosion/rust in bearings as well as in the tubular steel frame.

In swimming baths and bathrooms where strong gasses may be present, the transfer platform is particularly exposed to corrosion and should always be removed from such places after use.

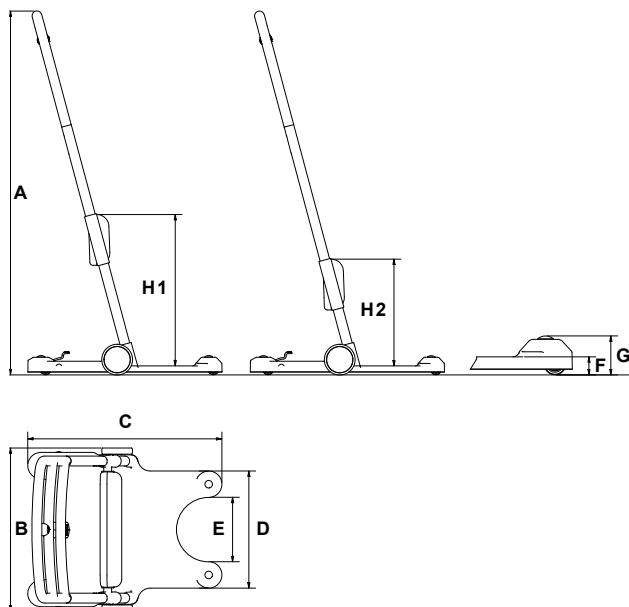8.00 Technical specifications

Functions

Safe working load (SWL)	175 kg
Entry height	30 mm
Handlebar	High friction, multi grip
Baseplate	High friction

Operations

Knee pad	Manual
Push-button	35 N
Brake, foot operated	85 N

Weight

Total	13 kg
Baseplate	8 kg
Ladder	5 kg

Materials

Aluminum, stainless steel, polyurethane foam, plastic

Measurements

A	1182 mm
B	530 mm
C	630 mm
D	380 mm
E	209 mm
F	30 mm
G	63 mm
H1	520 mm
H2	347 mm

9.00 Labelling

Pictograms/labels used on product

Read the manual before use

CE-marking

Medical Device Class I in accordance with EU MDR Regulation

Machine washing


UK Responsible Person

European Device Solutions Ltd. 15 Coanwood Drive, Whitley Bay, Tyne & Wear, NE25 9GB, United Kingdom.
Email: info@europeandevicesolutions.co.uk
Tel: +44-754-559-5464

Examples of labels

Batch number label

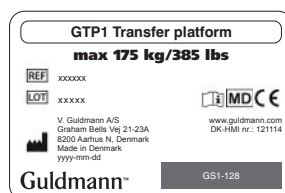Tube labels

10.00 EU-Declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

Where appropriate, the product comply with DS/EN ISO 21856 Assistive products – General requirements and test methods.

11.00 Environmental policy statement – V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

12.00 Warranty and service conditions

A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B. Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl
Guldmann Inc.