



GB/US Bath chair and bath stretcher

User manual – vers. 101.00

MD **CE**

Guldmann™

Bath Chair and Bath Stretcher

Item nos.

14600 - Mobile Bath Chair, incl. oval hook

14610 - Mobile Bath Stretcher, incl. oval hook

1.00	Purpose and use	3
1.01	Manufacturer	3
1.02	Intended purpose	3
1.03	Area of use	3
1.04	Conditions of use	3
1.05	Important/warnings	4
<hr/>		
2.00	Unpacking bath chair	5
2.01	Functional description of the bath chair	6
2.02	Marking for bath chair	7
<hr/>		
3.00	Unpacking bath stretcher	7
3.01	Functional description of the bath stretcher	8
3.02	Marking for bath stretcher	9
<hr/>		
4.00	Maintenance	9
4.01	Cleaning and maintenance	9
4.02	Replacement of components	9
4.03	Service life	9
4.04	Storage and use environment	10
<hr/>		
5.00	Technical specifications	10
<hr/>		
6.00	EU declaration of conformity	10
<hr/>		
7.00	Environmental policy statement - V. Goldmann A/S	10
<hr/>		
8.00	Warranty and service conditions	11
A.	Warranty	11
B.	Service or Repair	12

1.00 Purpose and use

1.01 Manufacturer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tlf. + 45 8741 3100
www.guldmann.com

1.02 Intended purpose

The Bath chair and bath stretcher is intended for lifting and transferring a person with disabilities in a seated or horizontal position.

1.03 Area of use

The Bath chair and Bath stretcher is suited for use in institutions, rehabilitation centers and in swimming pools.

1.04 Conditions of use

The Bath chair and Bath stretcher is designed for use with ceiling hoist systems and it is ideal in connection to the bath chair and bath stretcher allows for the lifting routines to be carried out in an ergonomically environment thus avoiding heavy lifting.

The Guldmann bath chair and bath stretcher is suited for lifting one person in to and out of swimming pools and similar, in a seated position (bath chair) or in a horizontal position (bath stretcher) and to facilitate the lifting situations, ensuring maximum safety when bathing.

The detachable and turnable armrests on the bath chair ensures a safe and easy transfer to another seat.

The integrated oval hook on the bath chair and bath stretcher makes the lifting out of and into the bath easy. Lifting only requires the snap hook of the ceiling hoist to be connected with the oval hook.

Important/warnings

Always exercise caution when using the bath chair or bath stretcher. This means that helpers should always make sure that they, the user and any other persons are safe in order to avoid injuries or accidents.

The bath chair and bath stretcher are designed for use in bathing environments and swimming pools with the following maximum values of:

Water quality:

pH	7,6
Free chlorine, mg/l	2,0
Bound chlorine, mg/l	0,5

Air humidity:

The relative air humidity in the room should be approx. 70% at an outdoor temperature of approx. 20°C and approx. 50% at an outdoor temperature of approx. 0°C.

- Please read the user manual before starting the use of the bath chair or bath stretcher.
- The bath chair and bath stretcher must only be used by trained personnel.
- The helper must pay attention to the well-being of the user when using the bath chair or bath stretcher.
- Do not exceed the recommended safe working load of the bath chair or bath stretcher.
- Products marked with the lowest lifting capacity determines the maximum load of the entire system.
- If a defect appears during use of the bath chair or bath stretcher, stop using it and contact the Guldmann Service Team for repair.
- The user on the bath stretcher may not turn or move in the lengthwise direction when lying in the lifter.
- To avoid unintentional movements of the user, the helper must position the hoist centrally over the user before lifting is started.
- Do not modify this equipment without the authorization of the manufacturer
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority.

The bath chair and bath stretcher must be cleaned and maintained according to the instructions mentioned in this manual, see section 4.01.


Visual inspection of the bath chair

If the bath chair is thought to be damaged upon receipt, it must not be used before it is checked and approved by a qualified person or the Guldmann Service Team.

Before use it is recommended to spray the product with a mineral oil / liquid paraffin, followed up by wiping up by wiping with a clean lint-free cloth.

The contents of the bath chair box


- Bath chair
- Oval hook
- User manual

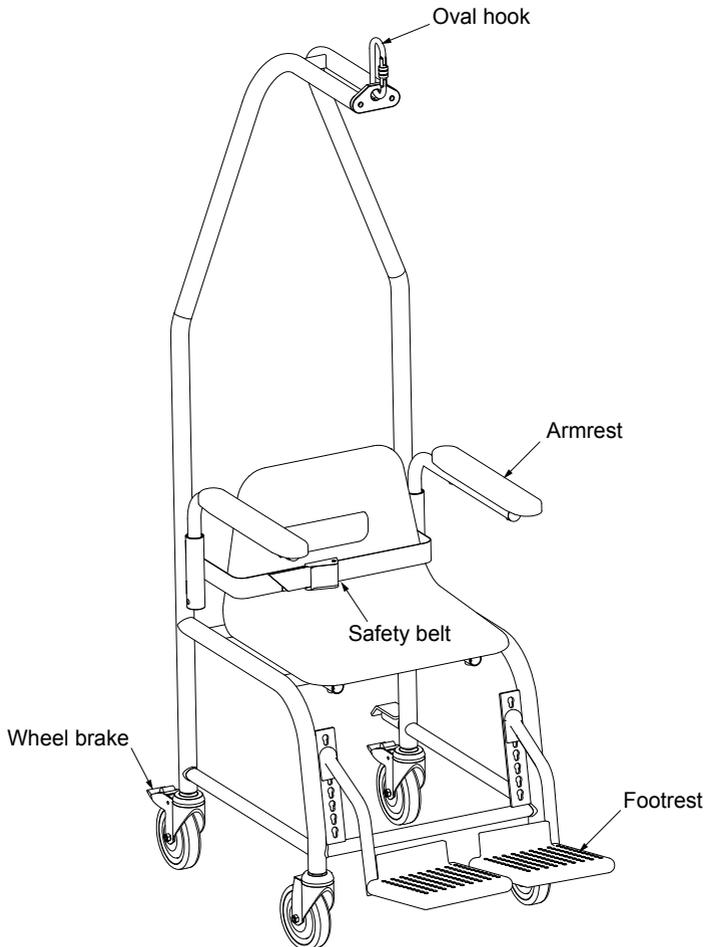


2.01

Functional description of the bath chair


The bath chair must be securely attached to either snap hook (GH2) or Oval hook (GH3), make sure that the hook is properly closed.

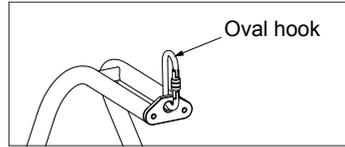


Footrests must be adjusted to the user, to relieve weight of users legs, and to ensure that legs stay in place during transfer.

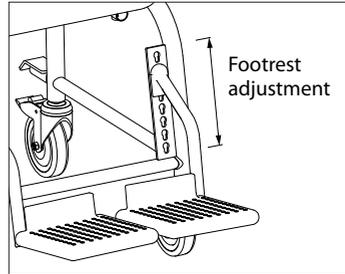

Armrests can be turned away or completely removed to allow easy access for user and helper during transfer between bath chair and e.g. Wheel chair.

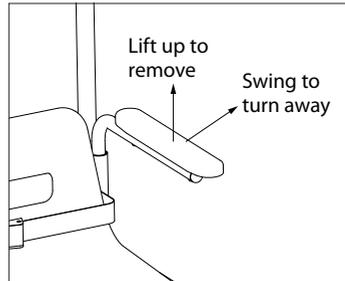

The bath chair is fitted with brake mechanism on the two back wheels, disabling the bath chair from moving around when transferring the user between bath chair and e.g. Wheel chair, when locked.

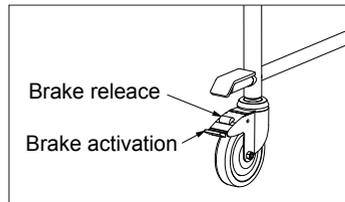

User must at all time during transfer be strapped in with the safety belt.



3.00

Unpacking bath stretcher


Visual inspection of the bath stretcher

If the bath stretcher is thought to be damaged upon receipt, it must not be used before it is checked and approved by a qualified person or the Guldmann Service Team.

Before use it is recommended to spray the product with a mineral oil / liquid paraffin, followed up by wiping with a clean lint-free cloth.

The contents of the bath stretcher box


- Bath Stretcher
- Oval hook
- User manual

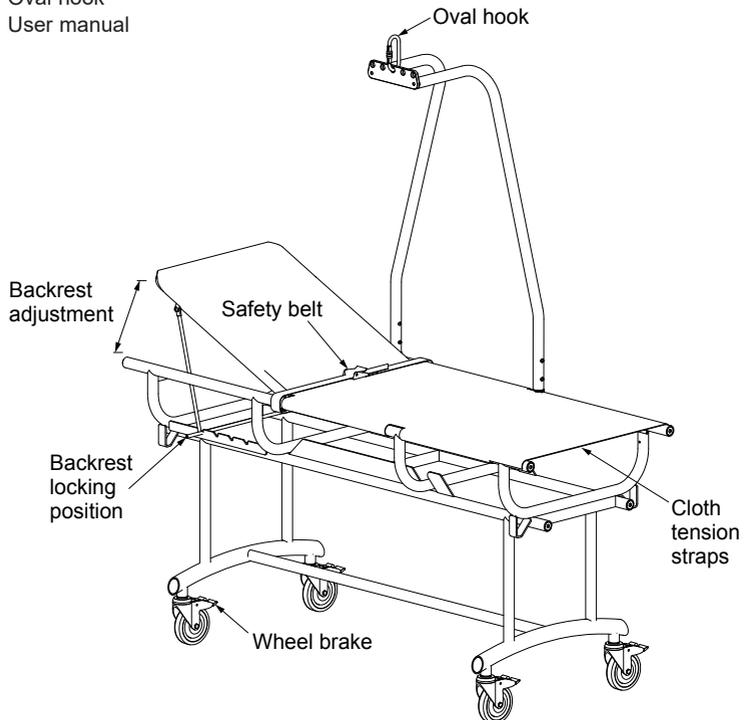


3.01

Functional description of the bath stretcher


The bath stretcher must be securely attached to either snap hook (GH2) or Oval hook (GH3), make sure that the hook is properly closed. The bath stretcher is equipped with 5 holes to adjust center of gravity.

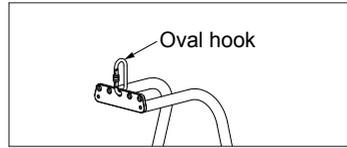

The backrest can be adjusted, raised and locked in 4 different positions.

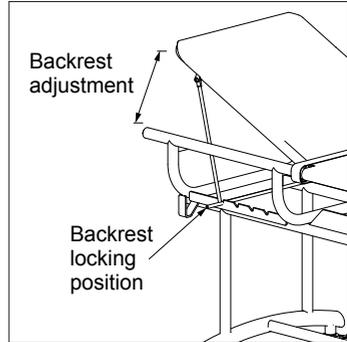

Stretcher cloth is fitted with straps, that allows the cloth tension to be adjusted, straps is located on back-side of cloth and can be tightened by pulling the strap end.

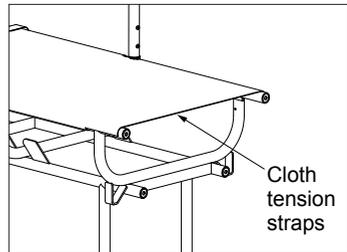

The bath stretcher is fitted with brake mechanism on all wheels, disabling the bath stretcher from moving around when transferring the user between bath stretcher and e.g. Wheel chair, when locked.

User must at all time during transfer be strapped in with the safety belt.

3.02 Marking for bath stretcher



4.00 Maintenance

4.01 Cleaning and maintenance

The bath chair and bath stretcher is made of materials with a high resistance to salts and chlorine. However, to avoid corrosion and discoloration, cleaning and maintenance is very important.

The below guidelines must be followed to ensure that the bath chair and the bath stretcher are nice to look at and have a long service life.

- Daily washing with clean water, followed by wiping with a firmly wrung cloth or wash leather.
- Washing with universal sanitary cleaning detergents once a week, followed up by spraying the surface with a mineral oil / liquid paraffin at a distance of approx 20 cm. Wipe with a clean lint-free cloth.

Particularly, during the first month of use, daily cleaning and wiping is very important where after the stainless steel will generate its own protecting film.

4.02 Replacement of components

Components may be replaced only by a qualified service technician or the Guldmann Service Team.

4.03 Service life

The bath chair and bath stretcher has an expected service life of 10 years, when subjected to continuous maintenance and correct use. In accordance with the international standard EN/ISO 10535 "Hoist for the transfer of disabled persons – Requirements and test methods" an inspection should be performed on the hoist at least once a year.

Guldmann recommends that regular safety/service inspection is performed at least once a year with regard to the pattern of usage.

Inspection of the bath chair and bath stretcher must be performed by a qualified service technician or the Guldmann Service Team.

In connection with the purchase of the bath chair and bath stretcher, Guldmann may offer a service agreement for this inspection.

4.04 Storage and use environment

The bath chair and bath stretcher are designed for use in Swimming pool areas.

Temperature +5 to 40 C°.

Keep the bath chair and bath stretcher in a dry room for long-term storage, according to the instructions mentioned in section 1.03 "Important/warnings".

5.00 Technical specifications

Max. load 255 kg. (560 lbs.)

Materials

Frame Stainless steel
Seat Molded polyester
Armrest Molded polyester
Wheels and bearings Stainless steel
Bolts Stainless steel
Footrest Painted, molded aluminum
Stretcher cloth Nylon

6.00 EU declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

7.00 Environmental policy statement - V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and

from any and all claims or liability arising as a result of equipment malfunction or misuse.

B.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

| Time to care |

V. Guldmann A/S
Corporate Office:
Tel. +45 8741 3100
info@guldmann.com
www.guldmann.com

Guldmann Inc.
Tel. 800 664 8834
Tel. 813 880 0619
Fax 813 880 9558
info@guldmann.net
www.guldmann.net