



Meet Guldmann's Smart Loops !

Patent Pending

Smart Loop Technology

By using modern weaving techniques to produce the “smart loop” sling strap, the product becomes homogeneous and even stronger as tolerances are improved by producing the entire sling strap and loops in one-piece.

Easy to use

- The simplified design with a one-piece woven strap and loops makes it easy to handle safely – even with one hand
- Colour coded, clearly visible from all angles, makes it easy to attach the sling correctly to the hanger

- Less bulk – when placing and removing the sling straps the risk of shearing against skin is significantly reduced; also a compact profile will cause less pressure if strap is mistakenly beneath the fabric

Improved cost of ownership and easier to store

- Less material and lower weight (1 kg per 10 slings equal to 7% of a repositioning sling) equals a lower cost of cleaning and laundry cycle transportation
- Improved folded profile, takes up less storage space

Safe to use

- Reduces the risk of unintentional entanglement in the surrounding environment (wheelchair, bed, etc.) because the loops are smoothly integrated into the strap
- Reduces the trip hazards as unused loops are a straight sling strap
- Easy to tuck away when not in use



Smart Loops Strap
One piece woven strap and loops

Standard Strap
Sewn on loops strap



Smart Loops

Equivalent features and function which will feel familiar and perform consistently if not better:

- Distinct and separate loops – same
- Distinct color differentiators sets loops apart- same solid grey, solid green, solid blue, green stripe, blue stripe
- The Smart Loop Strap has the same length and position of loops as the standard strap

Improvements

- Smart Loops less likely to get caught on equipment or be a hanging trip hazard
- Takes up less space when in use and in storage on shelf
- Weighs less overall – reduces laundering & operational expenses

Tips

- When newer, the straps' Smart Loops will be more closed; over time and with use, the Smart Loops will be slightly more open in resting position
- To open the Smart Loop, place two fingers inside the loop and spread the loop to assist in placing on the hanger hook

Time to care

V. Guldmann A/S
Tel. +45 8741 3100
info@guldmann.com
www.guldmann.com