
DK	Planløftebøjle, 375 kg	2
GB/US	Horizontal lifting support, 375 kg (825 lbs).....	11
DE.....	Liegebügel, 375 kg.....	21
SE.....	Planlyftbygel, 375 kg.....	31
NO	Planløftebøyle, 375 kg	41
FR.....	Support allongé, 375 kg	50
IT	Barella per sollevamento orizzontale, 375 kg.....	60
NL.....	Horizontale tilondersteuning 375 kg	70

User manual – vers. 101.00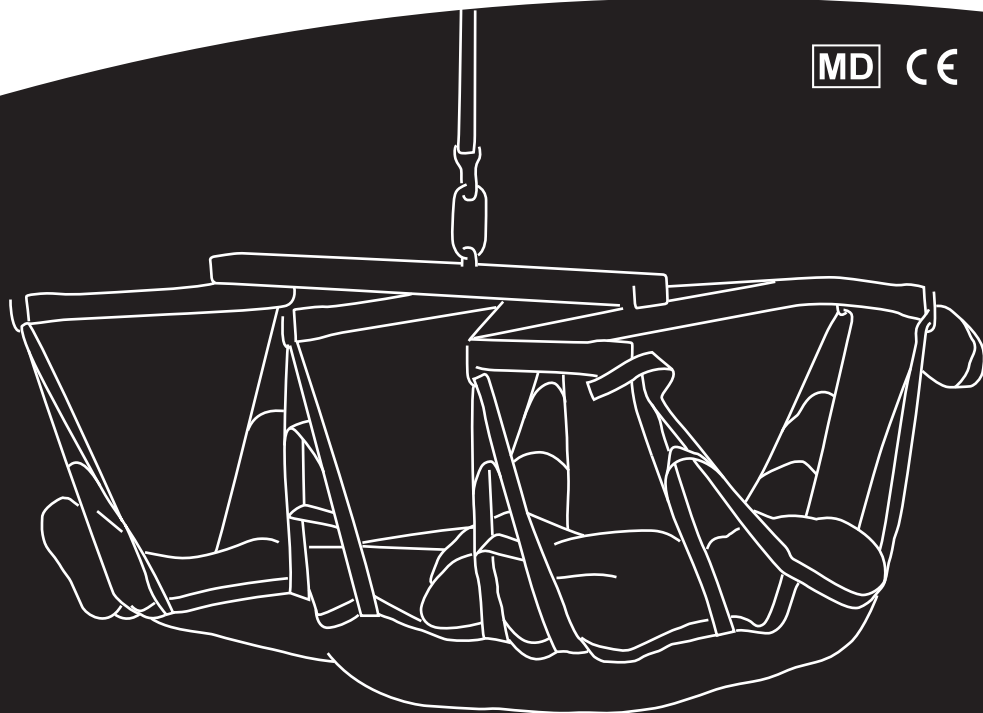

Guldmann™

DK Planløftebøjle, 375 kg

Vers. 101.00

Varenr.:

28456 – med trineløs tyngderegulering

28466 – med punktvis tyngderegulering

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GB/US . . . Horizontal lifting support, 375 kg (825 lbs)

Vers. 101.00

Item nos:

28456 – with continuously variable weight adjustment

28466 – with 10 different lifting positions

1.00	Purpose and use	12
1.01	Manufacturer	12
1.02	Intended purpose	12
1.03	Area of use	12
1.04	Conditions of use	12
1.05	Important/warnings	12
1.06	Unpacking	13
1.07	Horizontal sling	13
2.00	Description of features	14
2.01	Labels and Marking	14
2.02	Functional description	15
2.03	Accessories	17
3.00	Maintenance	17
3.01	Cleaning and Maintenance	17
3.02	Replacement of components	18
3.03	Service life	18
3.04	Storage and use environment	18
3.05	Disposal of the Horizontal lifting support	18
4.00	Technical specifications	18
5.00	EU declaration of conformity	18
6.00	Environmental policy statement – V. Guldmann A/S	19
7.00	Warranty and service conditions	19
A.	Warranty	19
B.	Service or Repair	20

1.00	Purpose and use
1.01	Manufacturer V. Guldmann A/S Graham Bells Vej 21-23A DK-8200 Aarhus N Tel. +45 8741 3100
1.02	Intended purpose The Lifting hanger is intended for lifting and transferring a person with disabilities in a horizontal position.
1.03	Area of use The Horizontal lifting support is suited for use in hospital, a nursing home and in institutions.
1.04	Conditions of use The horizontal lifting support is designed for use with both mobile lifters and ceiling hoist systems. It is ideal for moving a person in a horizontal position. The Horizontal lifting support may be used only for horizontal relocation* of a person lying on a level, horizontal surface. A level surface can be, for example, a bed, a stretcher or the floor. <i>* A horizontal relocation is defined as a relocation of a person lying on a horizontal surface where the horizontal position is kept during a vertical, horizontal or combined movement. The surface can be completely level or at an angle suited to the respective situation.</i> The use of the Horizontal lifting support is subject to the following: • The Horizontal lifting support is used by trained personnel. • The helper pays attention to the well-being of the user when using the Horizontal lifting support. • The Horizontal lifting support may be used only with a mobile lifter or a ceiling hoist and rail systems which have been tested and approved according to Guldmann's instructions.
1.05	Important/warnings Always exercise caution when using the Horizontal lifting support. This means that helpers should always make sure that they, the user and any other persons are safe in order to avoid injuries or accidents. • Please read the user manual before starting the use of the Horizontal lifting support. • Do not exceed the recommended safe working load of the Horizontal lifting support.

- Products marked with the lowest lifting capacity determine the maximum load.
- Use the Horizontal lifting support only to move a person "in a horizontal or angled position".
- If a defect appears during use of the Horizontal lifting support, stop using it and contact the Guldmann Service Team for repair.
- The user in the Horizontal lifting support must always be under supervision and may not turn or move in the lengthwise direction when lying in the lifter.
- To avoid unintentional movements of the user, the helper must position the hoist and the Horizontal lifting support centrally over the user before lifting is started.
- When the Horizontal lifting support is lowered with a user in it, the Horizontal lifting support may never rest on the user.
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority.
- Do not modify this equipment without the authorization of the manufacturer".

1.06 **Unpacking**

Visual check of the Horizontal lifting support

If the Horizontal lifting support is thought to be damaged upon receipt, it must not be used before it is checked and approved by a qualified person or the Guldmann Service Team.

The contents of the box

- Horizontal lifting support
- Oval hook

1.07 **Horizontal sling**

When working with a Horizontal lifting support

Attention should be paid to the functional level of the user and the work technique of the helper. The user should be lifted only to the extent that she/he is clear of the surface and then moved at this height.

The Horizontal lifting support is to be used with a lifting sling with eight straps (item number 28463) that are attached to hooks on the Horizontal lifting support. For proper fitting and use of the lifting sling, please refer to the product manual for this sling.

Warning!

Please exercise caution when positioning the sling straps on the Horizontal lifting support. Verify that the sling straps have been correctly positioned onto the hooks in the Horizontal lifting support. When pressing the arrow up button on the hand control you should verify once more that the sling straps remain in the correct position in the horizontal lifter hooks (Fig. 1).


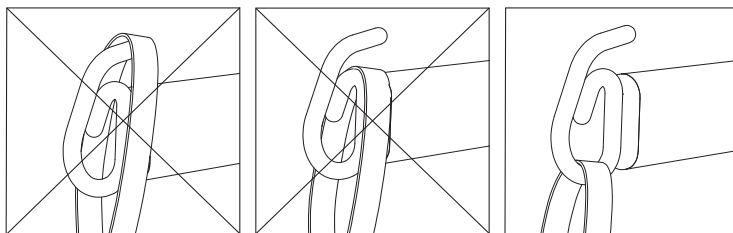
Fig. 1

Horizontal sling from another manufacturer

Guldmann disclaims any responsibility for defects or accidents which may occur when slings or horizontal slings made by other manufacturers are used. In case of doubt concerning the selection or use of a horizontal sling, please contact your supplier. Guldmann disclaims any responsibility for defects or accidents which may occur as a result of incorrect use of the Horizontal lifting support and horizontal sling, or because of lack of attention on the part of the caregiver or the user.

The Horizontal lifting support must be securely attached to either snap hook (GH2) or Oval hook (GH3 & Mobile lifter), make sure that the hooks are properly closed.

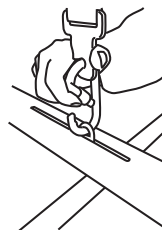2.00

Description of features

2.01

Labels and Marking

CE marking

Medical Device Class I in accordance with EU MDR Regulation

Read the manual before use

ID number label


2.02


Functional description

1. Horizontal lifting support with continuously variable weight adjustment
2. Horizontal lifting support with 10 different lifting positions
3. Oval hook
4. Weight adjustment handle
5. Swivel link GH2/GH3, spare part (Not included)

1. Horizontal lifting support with continuously variable weight adjustment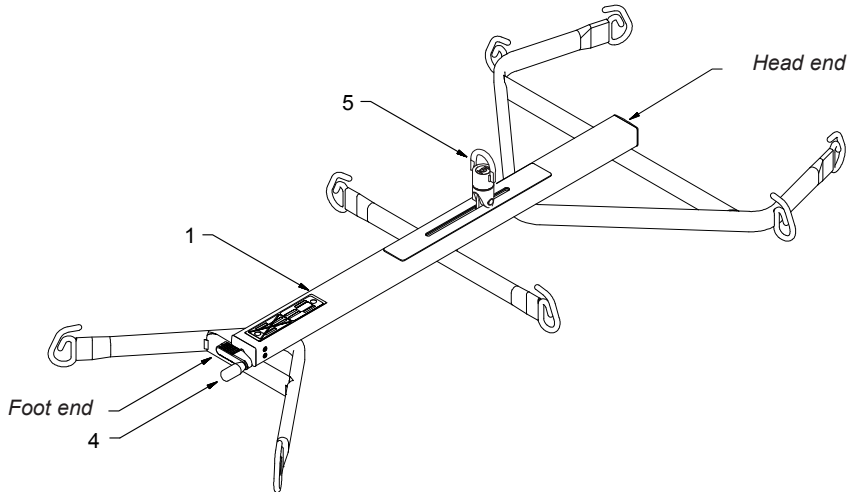

2. Horizontal lifting support with 10 different lifting positions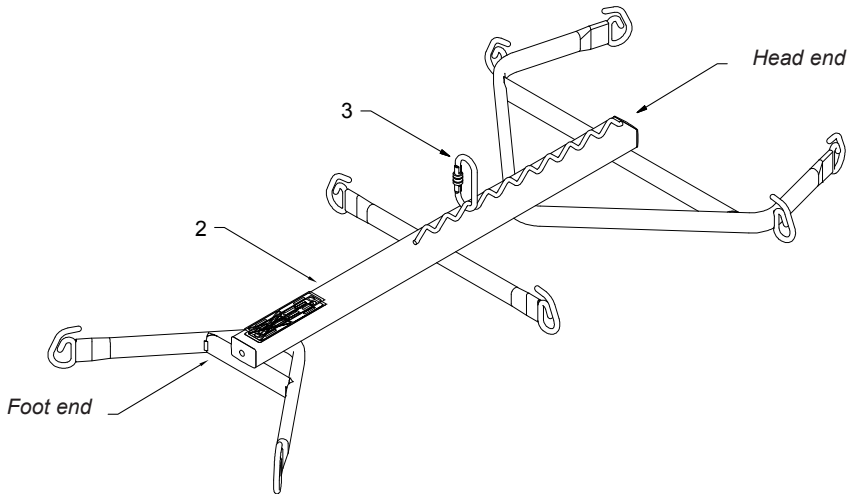

Attach the Horizontal lifting support to the hoist with the oval-shaped hook (pos. 3) or with the swivel (pos. 5).

On the model with continuously variable weight adjustment, the weight adjustment feature can be activated by turning the hand grip (pos. 4) clockwise to lower the head end and counterclockwise to raise the head end.

On the model with 10 different lifting positions, the weight can be adjusted by moving the hook/swivel to another suspension point – closer to the head end to raise the head end and closer to the foot end to lower the head end.


Horizontal lifting support used with Twin hoist

Installation of Horizontal lifting support

Place each of the lifting straps' oval hooks in the first eye of the Horizontal lifting support, one on each side.

The Horizontal lifting support can be aligned to the horizontal plane by adjusting the twin hoist's tilt function. For more information see the twin hoist manual.

Important
Check that none of the lifting straps are twisted after the installation of the Horizontal lifting support.

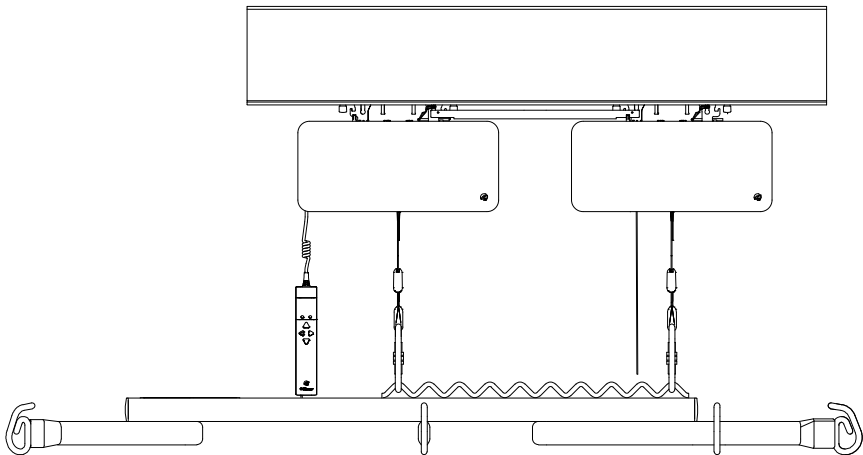2.03 Accessories

Horizontal sling for Horizontal lifting support	Item no. 28463
Oval hook	Item no. 21160
Swivel link GH2/GH3, spare part	Item no. 284520
Wall mounted parking space for Horizontal lifting support	Item no. 556968

3.00 Maintenance

3.01 Cleaning and Maintenance

The Horizontal lifting support is made of powder coated steel and resistance materials.

To ensure a proper appearance and long lifetime of the horizontal lifting support, it is recommended to follow these guidelines:

- Ordinary cleaning of the Horizontal lifting support is conducted with a damp cloth and ordinary detergent.
- Cleaning with strong detergents may only be applied on the Horizontal lifting support's exterior with a damp cloth.
- Do not use an autoclave.

The Horizontal lifting support does not need any additional maintenance apart from ordinary cleaning.

3.02	Replacement of components Components may be replaced only by a qualified service technician or the Guldmann Service Team.
3.03	Service life The Horizontal lifting support has an expected service life of 10 years, when subjected to continuous maintenance.
3.04	Storage and use environment The Horizontal lifting support is designed for indoor use. Temperature +5 to 40 C° Humidity <95% (not condensing) Keep the Horizontal lifting support in a dry room for long-term storage. Guldmann recommends to store the Horizontal lifting support on a wall mounted parking space when not in use.
3.05	Disposal of the Horizontal lifting support Local and national regulations on environmentally correct recycling must be observed.

4.00	Technical specifications Horizontal lifting support with continuously variable weight adjustment Lifting capacity max. 375 kg (825 lbs) Weight 11,6 kg (25.5 lbs) Horizontal lifting support with 10 different lifting positions Lifting capacity max. 375 kg (825 lbs) Weight 10,9 kg (24 lbs) Dimensions Horizontal lifting support (W x H x L). 716 x 110 x 1425 mm Materials Horizontal lifting support Powder coated iron & Plastic LDPE
-------------	---

5.00	EU declaration of conformity The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.
-------------	--

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

A.**Warranty**

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided

by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl
Guldmann Inc.