
DK	Stepless Lite ramper	2
GB	Stepless Lite Ramps	11
DE.....	Stepless Lite Rampen	20
SE.....	Stepless Lite ramper	29
IT	Rampe Stepless Lite	38
FR.....	Rampes Lite Stepless	47

Vers. 101.1

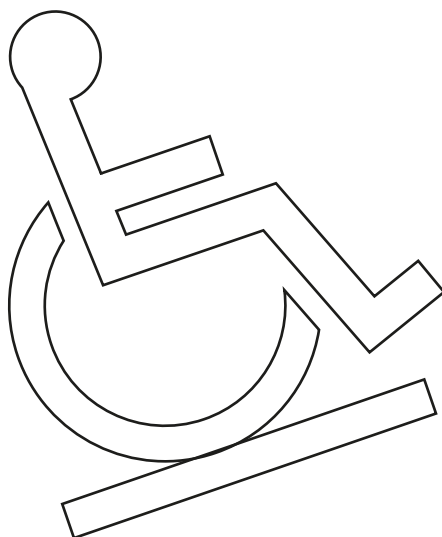DK Stepless Lite ramper

Varenummer
30100-xxx

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GB/US . . . Stepless Lite Ramps

Item no.
30100-xxx

1.00	Purpose and Application	12
1.01	Manufacturer	12
1.02	Intended Purpose	12
1.03	Area of Use	12
1.04	Conditions of Use	12
1.05	Important / Warnings	13
1.06	Unpacking and Preparation of the Ramp	14
2.00	Labels and Marking	14
3.00	Maintenance and Storage	15
3.01	Cleaning the Ramp	15
3.02	What Maintenance is Required By the Owner	15
3.03	Disposal	15
4.00	Service and lifetime	15
4.01	Lifetime	15
4.02	Safety/service inspections	16
5.00	Technical Specifications	16
6.00	EU declaration of conformity	18
7.00	Environmental policy statement - V. Guldmann A/S	18
8.00	Warranty and service conditions	18
A.	Warranty	18
B.	Service or Repair	19

1.00**Purpose and Application**

1.01**Manufacturer**

Sumitomo Rubber Industries, LTD
1-1, 2-Chome, Tsutsui-cho, Chuo-ku, Kobe
651-0071, Japan
Tel. +81 78 265 5708
Fax +81 78 265 5686

Authorized Representative/Importer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Denmark
Tel. +45 8741 3151
www.guldmann.com

1.02**Intended Purpose**

The Stepless Lite Ramps are intended for helping wheelchair users overcome height differences in the physical surroundings.

1.03**Area of Use**

The Stepless Lite ramps are suited for use wherever there is a need for wheelchair access past doorsteps and similar obstacles or from ground level to vehicle.

1.04**Conditions of Use**

Before use, check that there are no sand or other impurities on the surface. Also check that there is no other form of damage to the product. If there is sand or oil on the surface, this presents a dangerous situation as the user may slip. Remove any such object, incl. possible ice or snow before use.

The warning label "Top" is placed on the back indicating the direction of your highest point. Do not mistake the top for the bottom.

The upper surface has an anti-slip treatment. Do not mistake the upper surface for the back.

The ramp is folded when stored. Before use, unfold the ramp and place it on a firm underlay. The ramp should be positioned securing that the entire contact surface of the ramp is supported. While opening and closing the ramps, hold the ramps away from the body.

To facilitate the handling and the transport of the 70 cm ramp a handy carrying bag can be placed on the backside of the wheelchair to carry the ramp.

1.05 Important / Warnings

- Read the user manual carefully before using the ramp
- The ramps are only to be used by wheelchair users, users with walkers as well as users with baby carriages and must not be used for other purposes.
- The maximum load of the ramp is totally 300 kg (user + helper).
- Avoid shock loading.
- Only the user, the caretaker, or the luggage carrier are permitted to enter the ramp.
- Wheelchairs should be driven up and down at low speed – without a run-up.
- The ramp should be positioned securing that the entire contact surface is supported.
- The ramp should be used on a firm underlay.
- Move within the yellow lines and be careful not to drive the wheels off the ramp.

- When the ramp is carried, be careful not to hit against any objects.
- If the footrest or anti-tip tube of the wheelchair are not adjusted properly, they may cause damage to the anti-slip of the ramp. Be careful not to touch the surface during the adjustment and when driving on the ramp.
- The ramp should not be used for gradients greater than 1:5 - for electric wheelchairs or wheelchairs pushed by a helper.
- The ramp should not be used for gradients greater than 1:7 - for manual wheelchairs.
- Store the ramp indoors after use.
- To avoid braking, the ramp should be carried by its handles.
- When not in use the ramp must be stored upright. If the ramp is lying flat on the floor, people can step/jump on it and the ramp may be damaged.
- While opening and closing the ramps, hold the ramps away from the body.
- **WARNING:** If damage is suspected, the ramp must not be used before the dealer has authorised it.
- Any serious incident that occurred in relation to this device should be reported to the manufacturer and the local competent authority, if applicable.

1.06 **Unpacking and Preparation of the Ramp**

Visual Check of the Ramp

If the packaging is damaged upon receipt, the ramp should be inspected carefully for visible deficiencies, defect or faults. If damage is suspected, the ramp must not be used before the dealer has authorised it.

2.00 **Labels and Marking**

CE marking

Medical Device Class I in accordance with EU MDR Regulation

Read the manual before use

Caution

Product label

max 300 kg/660 lbs

cm / inch	1:5 	1:7 
50 cm / 20"	10 cm / 4"	7 cm / 2 1/4"
100 cm / 40"	20 cm / 8"	14 cm / 5 1/2"
150 cm / 60"	30 cm / 12"	21 cm / 8 1/2"
200 cm / 79"	40 cm / 15 1/4"	29 cm / 11 1/2"
250 cm / 98"	50 cm / 19 1/2"	36 cm / 14"
300 cm / 118"	60 cm / 23 1/2"	43 cm / 17"

US: To apply to ADA codes use 1:12

Serial number label

REF

SN

V. Guldmann A/S
Graham Bells Vej 21-23A
8200 Aarhus N
Denmark
Tel. +45 8741 3100
www.guldmann.com

Sumitomo Rubber Industries Ltd.
2-1-1 Tsutsui-cho
Chuo-ku, Kobe, Hyogo
651-0071, Japan
Tel. +81-79-458-8453
www.srigroup.co.jp

3.00 Maintenance and Storage

3.01 Cleaning the Ramp

Clean the ramp with a damp cloth, sponge or soft brush applied with a normal cleaning agent. The ramps may also be rinsed under running water from garden hoses or the like. Chemicals, strong acid or strong alkali should not be used to clean the ramp.

3.02 What Maintenance is Required By the Owner

If the ramps are slippery when covered with ice, use warm water to take the ice off.

If there is sand or oil on the surface, it should be cleaned well before use.

3.03 Disposal

Local and national regulations on environmentally correct recycling must be observed.

4.00 Service and lifetime

4.01 Lifetime

Given the maintenance prescribed above, the ramps will have a useful life of approx. 5 years.

© Guldmann 05/2021 # 561169_101.1

15

Safety/service inspections

Parts which are worn or defective should be replaced with new spare parts.

1. Visual check of the product

- Check the product for wear and tear.
- Check the product for any faults.
- Check that there is no other form of damage to the product.

2. Testing of the product, as for normal use

- Check all the product's functions, loaded and unloaded.

3. Control of the mechanical state of the product

- Clean dirt and other impurities from the product.
- Inspect and assess vital parts of the product.
- Replace any parts of the product which are defective or worn.

4. Go through point 2 again to recheck

5. Did any new problems arise/were new problems noted in point 4?

- If new problems arose, go back to point 3.
- If no new problems were noted, conclude the inspection.

Technical Specifications

Stepless Lite Ramps

Material

Panel	Graphite fibre/Glass fibre
Joint	Nylon
Handle	Nylon
Corner protector	Rubber
Walk surface	Rubber tip paint
Paint	Urethane

Colour

Walk surface	Grey
Center Line	Blue
Side Portion	Yellow

Durability

Max load	300 kg
----------------	--------

Stepless Lite ramps 30100-205

Structure

Length	205 cm
Width (in use)	39 x 2 = 78 cm
Edge barrier	Height 25 mm
Thickness (Fold)	106 mm
Max height difference	.41 cm (1:5)
Weight kg	9,5

Stepless Lite ramps 30100-165

Structure

Length	165 cm
Width (in use)	39 x 2 = 78 cm
Edge barrier	Height 25 mm
Thickness (Fold)	106 mm
Max height difference	.33 cm (1:5)
Weight kg	7,5

Stepless Lite ramps 30100-125

Structure

Length	125 cm
Width (in use)	39 x 2 = 78 cm
Edge barrier	Height 25 mm
Thickness (Fold)	106 mm
Max height difference	.25 cm (1:5)
Weight kg	6

Stepless Lite ramps 30100-085

Structure

Length	85 cm
Width (in use)	39 x 2 = 78 cm
Edge barrier	none
Thickness (Fold)	38 mm
Max height difference	.17 cm (1:5)
Weight kg	4

Stepless Lite ramps 30100-070

Structure

Length	70 cm
Width (in use)	36,5 x 2 = 73 cm
Edge barrier	none
Thickness (Fold)	38 mm
Max height difference	.14 cm (1:5)
Weight kg	3,5

Accessory: 30100-000 Stepless wheelchair carrying bag for 70 cm ramp

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

Guldmann continues to work towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmanns goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, as much as possible, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on

Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl