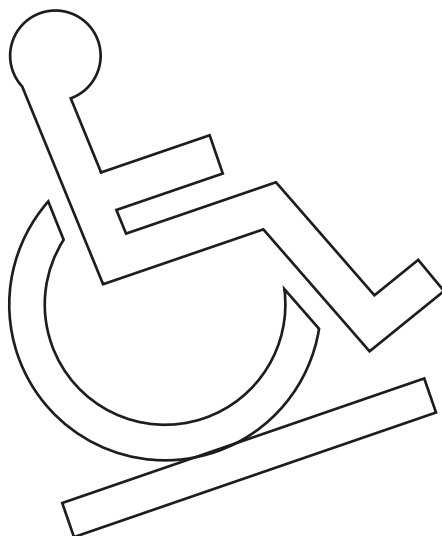
DK	Dækplader & dørtrinsramper	3
GB/US	Cover plates & doorstep ramps	9
DE	Abdeckbleche und Türschwellenrampen	15
SE	Täckplattor och tröskelramper	22

Vers. 3.00

I

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GB/US Cover plates and doorstep ramps

Product numbers	
780xxx	Cover plate
39167/77	Doorstep ramp

1.00	Purpose and use	10
1.01	Manufacturer	10
1.02	Purpose and use	10
1.03	Use	10
1.04	Important/warnings	10
1.05	Unpacking and preparation of the ramp/cover plate	11
2.00	Labels and Marking	11
3.00	Cleaning and maintenance	11
3.01	Cleaning the ramp and cover plate	11
3.02	Corrosion	11
3.03	Maintenance	12
4.00	Storage and stacking	12
5.00	Service and useful life	12
5.01	Useful life	12
5.02	Service	12
6.00	Technical specifications	12
7.00	EU Declaration of conformity	13
8.00	Environmental policy statement - V. Guldmann A/S	13
9.00	Warranty and service conditions	13
A.	Warranty	13
B.	Service or Repair	14

1.00	Purpose and use
1.01	Manufacturer V. Guldmann A/S Graham Bells Vej 21-23A DK-8200 Århus N Tel. +45 8741 3100
1.02	Purpose and use Purpose Stepless ramps are intended for helping wheelchair users overcome height differences in the physical surroundings. Application <i>The cover plate</i> replaces the doorstep. By using the cover plate, the wheelchair user can access one room from another regardless of the height difference. <i>The doorstep ramp</i> have ramps on both sides of the doorstep. The wheelchair user can therefore use the ramp to get from room to room.
1.03	Use Cover plate A doorstep creates an obstacle for wheelchairs, rollators and other mobility aids. This obstacle can be permanently removed by taking out the existing doorstep and installing a Stepless cover plate over the opening that is created as a result. The cover plate is tailored to the actual length of the installation site so that it fits exactly to the width of the door. Doorstep ramp The doorstep ramp is placed over the existing doorstep and offsets the height difference between two rooms. The doorstep ramp is not fixed and can therefore be easily removed after use.
1.04	Important/warnings • Read the entire user manual prior to putting the ramp/cover plate into operation. • The maximum load must not be exceeded. • Movement up and down the ramp must take place slowly – without run-ups. • The ramp must be positioned so that the entire contact surface is supported. • The ramp must be used on a firm underlay. • The cover plate must be completely supported and must not, for example, be used as a bridge between two rooms. • The ramp should not be used for gradients greater than 1:5 by electric wheelchairs or wheelchairs operated by caregivers. • The ramp should not be used for gradients greater than 1:7 for manual wheelchairs.

1.05 **Unpacking and preparation of the ramp/cover plate**

Visual check of the ramp/cover plate

If the packaging is damaged on receipt, the ramp/cover plate should be carefully checked for noticeable imperfections, defects or flaws.

If the ramp/cover plate is thought to be damaged, it must not be used until quality service technicians or the Guldmann Service Team have approved it.

2.00 **Labels and Marking**

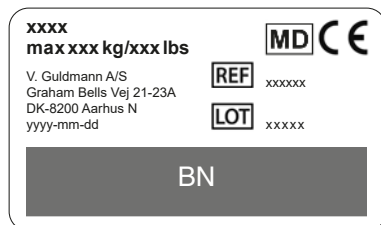
CE marking

Medical Device Class I in accordance with EU MDR Regulation

Product label

3.00 **Cleaning and maintenance**

3.01 **Cleaning the ramp and cover plate**

Clean the ramp and cover plate with a normal cleaning agent applied with a damp cloth, sponge or soft brush. The ramp and cover plate may be rinsed under running water. No chemicals or autoclave must be used to clean the ramp and cover plate.

3.02 **Corrosion**

The ramp and cover plate are made of corrosion-resistant materials and do not require further corrosion prevention.

3.03**Maintenance**

No particular requirements other than cleaning.

If the ramp or cover plate surface is covered with ice and/or snow, remove the ice/snow before use.

Note: Salt (NaCl) is corrosive to the ramps and cover plates and will damage them over time. Instead, use a salt substitute, such as magnesium chloride (MgCl₂). When using magnesium chloride to de-ice the ramps and cover plates instead of salt, you avoid oxidation and corrosion.

4.00**Storage and stacking**

No particular requirements.

5.00**Service and useful life**

5.01**Useful life**

Given the maintenance/cleaning prescribed, the ramp and cover plate will have a useful life of 15 years.

5.02**Service**

Service inspections must be carried out in accordance with national legislation.

Parts that are worn or defective must be replaced with new parts from V. Guldmann A/S.

1. Visual check of the product

- Check the product for wear and tear.
- Check the product for any faults.
- Check that there is no other form of damage to the product

2. Have new problems arisen/been found?

- If the product is damaged or parts need to be replaced, please contact the Guldmann Service Team or your distributor.

6.00**Technical specifications****Doorstep ramp (391xxx)**

Width x length x height 670x600x25 / 770x600x25 mm

Max. load 350 kg

Materials Bended 3 mm aluminium plate with black non-slip covering.

Cover plates (780xxx)

Lengths	2000 mm
Widths	80, 105, 130, 150 and 180 mm
Max. load	500 kg
Materials	Extruded aluminium with built-in non-slip protection (grooves)

7.00

EU Declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device Class I.

8.00

Environmental policy statement - V. Guldmann A/S

Guldmann continues to work towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmanns goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, as much as possible, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

9.00

Warranty and service conditions

A.

Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the

user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann Inc.
Guldmann GmbH
Guldmann Sverige AB