DK	Sling Sleeves.....	2
GB/US	Sling Sleeves.....	3
DE.....	Sling Sleeves.....	4
SE.....	Sling Sleeves.....	5
NO	Sling Sleeves.....	6
FR.....	Sling Sleeves.....	7
IT	Sling Sleeves.....	8
NL.....	Sling Sleeves.....	9

User manual – vers. 101.0

Guldmann™

DK... SLING SLEEVES

Vers. 101.0

Varenr.:
562495 Sling Sleeves

1.00 Formål og anvendelse

1.01 Producent

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. + 45 8741 3100
www.guldmann.com

1.02 Erklærede formål

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1.02 Intended purpose

The Sling Sleeves are used to provide extra protection around the sling leg straps when working with especially sensitive users. The Sling Sleeves must be applied to the leg straps before the straps are positioned under the user's legs.

2.00 Maintenance

2.01 Cleaning

Normal washing at the indicated temperature

Do not use bleaching agent

Tumble-drying at low temperature

Do not iron

2.02 The owner's daily maintenance duty

Check the **accessory** for wear and damage before use according to the following checklist which is not intended to represent all potential inspection steps. Potential damage may vary. Judgment of inspector/site prevails.

Accessory inspection checklist

Before using Guldmann accessory check the following:

Is the accessory clean?

Follow facility specific infection control procedure.

Is the accessory's label present, legible and complete?

Missing, illegible or incomplete **accessory** label(s) could make identification of appropriate size of the **accessory**, function of **accessory**, and or weight limit capacity of the **accessory** impossible.

Are the straps and stitches intact?

- Look for broken or worn stitches
- Look for knots in straps
- Look for tears or fraying of straps
- Look for snags or punctures or holes
- Look for any particles in fabric or straps

Is the fabric intact?

- Look for abnormal wear patterns, excessive wear, abrasive evidence
- Look for cuts or frayed fabric
- Look for unusual or significant discoloration
- Look for snags, punctures, tears, holes
- Look for frayed or insecure seams
- Look for any acid / caustic / thermal burns
- Look for changes in material consistency, e.g. increased stiffness
- Look for any imbedded particles

Has the shape of the accessory been altered, made shorter or longer in relation to the original size using knots, needles, tape or other methods?

Conclusion

If the **accessory** suffers from one or more of the above mentioned conditions then it must be taken out of service.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl
Guldmann Inc.