

Trouble Shooting Sheet

RESET


When you pull the red chord it releases the RESET button located on the lift. You will see a yellow ring around the button. This indicates that the lift has no power. When you want to reset the lift to operate you will have to manually push the RESET button back in.

EMERGENCY STOP

Pull the red strap once to immediately shut down the lift.

EMERGENCY LOWERING

Constant pull on the red strap will lower the patient in the event of hand control failure

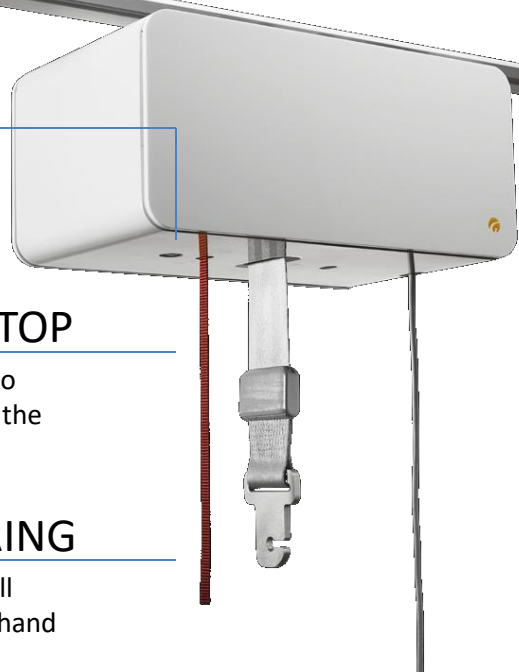


GREEN

Good to Go. No problems

YELLOW

- Low battery
- Not charging
- Weight overload
- Service required

Lamp indicator

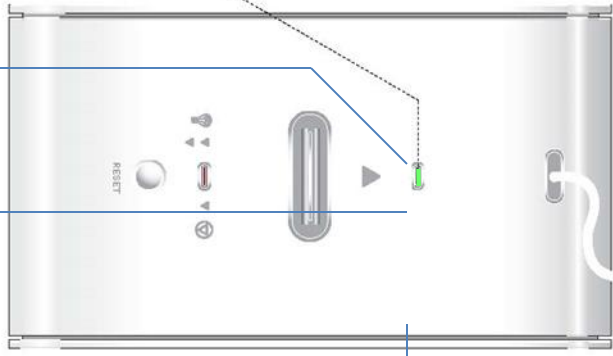

AUDIBLE BEEPING WARNINGS



- 3 x beeps indicates lift is not charging
- Beeps at button activation indicates lift fault
- Beeps at button activation if battery at critical level
- Beeps at button activation when weight overload

MECHANICAL LOWERING

Emergency use only open side cover of lift and turn the handle labeled EMERGENCY DOWN. This will slowly lower the lift in the event of an electrical failure.



How to use the Hand Control

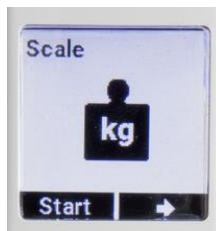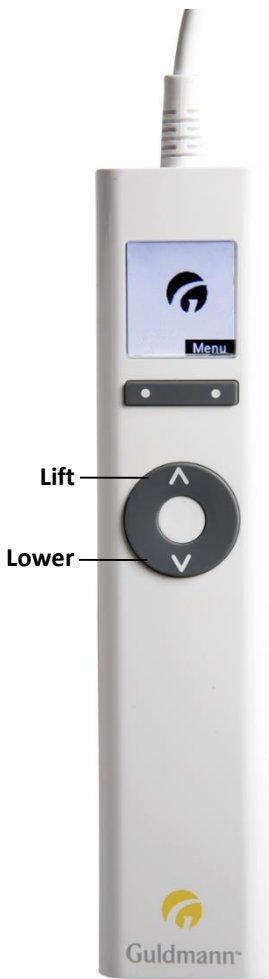

MENU

Guldmann logo displayed. This screen indicates lift is functioning and ready to go. Press the **NEXT** button to access modules available.

Possible Modules include **Scale, CLM, and Service.**

Set-Up functions include access to select language preference and select unit of measure for weighing in pounds or kilograms.

SCALE

A scale image will appear. To zero the scale in advance of a weighing task: lower the hanger to your working space, attach the patient's sling accessory, park hand control on the hanger, **push 'Start', followed by 'Zero' or 'Tare'; once '0.0' displays, push 'Exit'.** Then proceed with your patient care / **weighing task**; once the pt. is free of all surface/contact to **push 'Start' to display the weight.**

CLM

An image of the letter 'i' indicates you have accessed **the CLM program.** Press the **"SHOW"** button to access **data** such as lifetime lifts, heavy lifts, and weekly lifts; or proceed past CLM data by pressing the 'ARROW' button.

SERVICE

This screen will **indicate the due date of the next scheduled preventative maintenance.** If the lift has not been serviced prior to this date the lift will audibly remind user with a beep every time the lift is activated until it has been serviced.

WARNING DISPLAY

The hand control will display all warnings to the user with pop up text. When the lift motor displays the **YELLOW** light, the hand control **screen will display the reason** for the lift's yellow light in an error message - see some example warnings.

