



Guldmann™

## Guldmann GH3+ Trainer Module Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes knowledge of the patient overall weight, patient assistance level and how much weight should be offset in Trainer Module mode	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Demonstrates ability to activate the Trainer Module as indicated.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Demonstrates attention to guard the hand control and the hanger's stability when lowered and in proximity to patient / resident or colleagues before it is loaded with weight.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
5.	Verbalizes understanding of the 30 minute time bank available for Trainer Module mode.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Verbalizes understanding of time bank repletion rate.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Understands and demonstrates ability to perform quick stop function with handset during Trainer Module mode, in order to stop or 'arrest' the patient's position if loss of balance occurs	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
8.	Verbalizes understanding that a slow controlled lowering (no abrupt contact) will occur during Trainer Module mode, if hand control is not used to exit the mode.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Demonstrates ability to execute desired patient care task in Trainer Module mode.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Verbalizes awareness of and how to find resources materials to answer questions on operation, care, and service of Trainer Module	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	