



Guldmann™

Guldmann Sit On Comfort High Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes how to inspect the sling for damage, wear and tear, and what criteria indicates when it's time to remove a sling from service.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Verbalizes and demonstrates where to verify the weight capacity (SWL) of the Guldmann Sit On Comfort High sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Verbalizes and demonstrates how to select the appropriate size for the patient or resident				
5.	Verbalize understanding that the lowest weight capacity between equipment and sling will determine the safe working load for the task (i.e. if the hoist or hanger or sling or other accessory vary in SWL).	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Verbalizes and demonstrates how to both correctly align the Guldmann Sit On Comfort High sling to the patient, as well as applying on the patient in supine via rolling on horizontal surface	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Verbalizes understanding that loops of the Guldmann Sit On Comfort High sling need to match to its opposite side(s) (i.e. both shoulder straps should have the same color loop attached – both grey or both green; both leg straps should have the same color loop). It is acceptable to vary the color loop attached between the shoulders and legs.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	



Guldmann™

Guldmann Sit On Comfort High Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
8.	Demonstrates attention to ensure slings loops are securely on the hanger bar's hooks before it is loaded with weight and after patient is lifted/before moving away from surface.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Identifies the Sit On Comfort High features to place sling correctly: dart stitching at both hips, center contrast stitching	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Demonstrates how to use the head support feature of the Guldmann Sit On Comfort High sling should the patient required it	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
11.	Verbalize proper cleaning and facility laundering protocol for reusable slings, as applicable, and where clean slings are stored.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
12.	Verbalizes awareness of resource material to answer questions on sling use and functionality, manufacturer's recommendations for cleaning, and Inspection Guidelines / when to remove a Guldmann Sit On Comfort High sling from inventory.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	