



Guldmann®

Guldmann Sit On Comfort High Sling Quiz

1. There are no risks to the patient in using the wrong size of Sit On Comfort High sling.

- A. True
- B. False

Answer B

2. When is using a Sit On Comfort High sling appropriate?

- A. Lifting and transferring from all furniture surfaces
- B. Recover patient from floor when circumstances are appropriate
- C. Sitting balance, mobilization and hygiene positioning
- D. All of the above

Answer D

3. Is Sit On Comfort High sling appropriate for use with someone with no head control?

- A. Yes, Sit On Comfort High sling is for persons who require assistance with head control
- B. No, Sit On Comfort High sling is only used by users who can Sit on their own
- C. Yes, Sit On Comfort High provides full back height and a separate head support feature
- D. A and C

Answer D

4. What patient care tasks can you perform with use of the Sit On Comfort High sling?

- A. Usual seated transfers for persons with both lower extremities intact
- B. Seated transfers with persons with one or two below knee and above knee amputations, when assess for clinical appropriateness
- C. Seated transfers for persons requiring additional space between their thighs or have impairments in thighs' region or groin
- D. All of the above

Answer D

5. How do you know when a Sit On Comfort High sling should be taken out of service?

- A. One or more of the loops are knotted together
- B. The threads on a seam of the Sit On Comfort High sling are frayed
- C. The fabric is ripped or has a puncture hole
- D. All of the above

Answer D

6. What should you do with a soiled Sit On Comfort High sling?

- A. Reusable slings are discarded if they are very soiled
- B. When sling appears clean after one patient use, it can put back on the shelf for use
- C. Follow procedure provided by organization for soiled sling laundering
- D. All of the above

Answer: C



Guldmann®

Guldmann Sit On Comfort High Sling Quiz

7. When do you explain to patient why and how you will use the Sit On Comfort High sling?

- A. Only when you have time to give an explanation
- B. Only when there are family members in the room
- C. Only for patients that may complain about the use of slings
- D. Always explain the use of the equipment and sling every time you are using with the patient, and which task will be accomplished at the time of use

Answer: D

8. You need to use the Sit On Comfort High sling for a seated transfer but unsure how to select size and how to properly apply to patient.

- A. Ask your strongest co-worker to help manually transfer the patient or resident
- B. Ask your unit peer leader or a trained co-worker to assist you in using the Sit On Comfort High sling. Setup 'just in time' training with UPL or SPHM Coordinator to further review application and use of the Sit On Comfort High sling
- C. Apply it like a similar seated transfer sling from a different manufacturer, they have all the same instructions
- D. All of the above

Answer: B

9. You need a sling size that's not available in your unit's storage, how do you proceed?

- A. Take a Sit On Comfort High sling from another patient and use it for your patient
- B. Ask a co-worker to assist with a manual transfer
- C. Obtain a clean Sit On Comfort High sling in the size you need with approval of a nearby unit, or from where overstock clean slings are stored; then follow facility procedure to document insufficient access to needed supplies for patient(s) on your unit
- D. Wait until the size sling is available on the unit

Answer: C

10. Clean slings are already inspected for wear and damage, so no extra steps are needed before patient application.

- A. True, caregiver can trust that the sling meets manufactures recommendations after in-house or out sourced laundry has cleaned the sling
- B. False, a clean sling may not pass safety checklist, slings should be checked for overt damage with each patient application

Answer: B