



Guldmann™

Guldmann GH1 – Quiz Questions

1. A user activates the Guldmann GH1 hoist by:

- A. Moving the hanger and rails
- B. Pulling on the red cord
- C. Pressing an arrow button on the hand control
- D. All of the above

Answer C

2. When is using a Guldmann GH1 hoist appropriate?

- A. When patient is unable to reposition and roll themselves in bed
- B. When a patient is unable to stand and walk safely
- C. When a patient requires one or more limbs or body parts lifted and supported
- D. When patient needs to be transferred to a chair or bed
- E. All of the above

Answer E

3. How do you know the hoist requires service attention?

- A. Indicator light glows yellow amber
- B. Indicator light glows green
- C. Hoist will emit a low tone beep
- D. A and C

Answer D

4. How do you verify the weight capacity of the hoist?

- A. Manufacturer's labels on the hoist.
- B. Refer to the Product Manual
- C. Ask the patient/ resident
- D. Ask a co-worker

Answer: A

5. How do you engage an emergency stop?

- A. Through the hand control function
- B. Call BioMed department
- C. Pull the red cord
- D. All of the above

Answer: C



Guldmann™

Guldmann GH1 – Test Questions

6. How do you engage a manual lowering function?

- A. Through the hand control function
- B. Remove the sling and patient from the hanger
- C. Pull the red cord
- D. Pull and hold down the red cord until lowering is complete

Answer: D

7. How do you reactivate the hoist after emergency stop or manual lowering?

- A. Through the hand control function
- B. Push the Reset button up until slightly recessed and push button on hand control
- C. Remove the hanger from the lifting strap
- D. All of the above

Answer: B

8. How many persons should operate the Guldmann GH1 hoist?

- A. Minimum of 1 person
- B. Indications for 2 or more persons are patient and task dependent
- C. Indications for more than 1 person are determined by the facility and their governing rules and regulations
- D. All of the above

Answer: D

9. Which item is NOT a responsible caregiver use of the GH1 hoist?

- A. Guarding the hand control when not in use by parking it on the hanger
- B. Guarding the hanger when it is lowered and in proximity to patient/resident or co-workers when it is not stable and loaded with weight
- C. Allowing patient/resident and visitors to use the hoist without permission, instructions, supervision
- D. Referring to the Product Manual and Manufacturer's literature for guidance

Answer: C

10. What are proper step to ensure the GH1 hoist charges appropriately?

- A. Place hand control into the charging station
- B. Ensure the power supply is turned on
- C. Hear 3 beeps to verify the GH1 is charging
- D. All of the above

Answer: D