



Guldmann™

Guldmann GLS5.2 Active Lifter

Quiz Questions

1. When is using the GLS5.2 lift appropriate?

- A. When a powered mobile working tool is indicated
- B. When moving standing semi-independent persons
- C. Moving to and from toilet, commode chair, or bedside chair
- D. Practicing sit to stand movement and static standing
- E. All of the above**

Answer E

2. Which of the following are accurate features GLS5.2?

- A. Use either set of hooks on lifting arm to lift to standing
- B. Knee pad and foot plate must always be used
- C. Use all hooks on lifting arm to perform a low clearance seated lift with Active Micro Plus or Basic sling
- D. A and C**

Answer D

3. What is the SWL (safe working load) of the GLS5.2?

- A. 155 kg. / 341 lbs.
- B. 205 kg. / 450 lbs.**
- C. 254 kg. / 560 lbs.
- D. 175 kg. / 385 lbs.

Answer: B

4. When do you lock the GLS5.2?

- A. When lifting and lowering the patient
- B. When patients weight exceeds 275lbs
- C. When the GLS5.2 is parked and not in use**
- D. All of the above

Answer: C

5. What are user (patient) requirements for GLS5.2?

- A. User can balance in a sitting position
- B. User can bear weight through their leg(s)
- C. User can use at least one hand or arm to pull to standing position
- D. User can understand and follow directions
- E. All of the above**

Answer: E



Guldmann™

Guldmann GLS5.2 Active Lifter

Quiz Questions

6. Which are accurate statements regarding the GLS5.2?

- A. One should use the Hip Support with the GLS5.2 when extra user support is indicated for the patient buttocks during raising or lowering
- B. Use of the Hip Support is mandatory for persons greater than 275 lbs.
- C. Position the Hip Support over the upper third of the buttocks for raising or lowering
- D. A and C**

Answer: D

7. Which item is NOT a responsible caregiver use of the GLS5.2?

- A. Conducting a full risk assessment of user's functional communication and capability before use
- B. Explain to the user the use of and the procedural steps for using the GL5.1 in advance of and during transfers and movement
- C. Leaving the user unattended when GLS5.2 is in use**
- D. Referring to the Product Manual and Manufacturer's literature for guidance

Answer: C

8. It is possible to tell what percentage charge the GLS5.2 battery has.

- A. True**
- B. False

Answer: A

9. What is maximum recharging time for GLS5.2?

- A. 4 hours**
- B. 2 hours
- C. overnight
- D. 3 hours

Answer: A

10. Name two things a caregiver should always do when using the GLS5.2

- A. Confirm access to the slings designed for the Active Lifter
- B. Plug in the lift to start
- C. Ask a co-worker for assistance
- D. Use caution when approaching patient's feet with GLS5.2
- E. A and C
- F. A and D**

Answer: F