



Guldmann™ Guldmann Repositioning Sling Skills Check

	Key Element	Satisfactory	Unsatisfactory	Needs Practice	Comment or N/A
1.	Always explain procedure to patient or resident before and during task. When working with others during task, always communicate and verify actions.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2.	Verbalizes how to inspect the sling for damage, wear and tear, and what criteria indicates when it's time to remove a sling from service.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3.	Verbalizes and demonstrates where to verify the weight capacity (SWL) of the Guldmann Repositioning sling.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4.	Verbalizes different functions of the Guldmann Repositioning sling	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
5.	Demonstrate ability to use Guldmann Repositioning sling to lift up patient and reposition / boost up in bed	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6.	Demonstrate how to use the Guldmann Repositioning sling to turn a patient to sidelying position	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
7.	Demonstrates attention to stopping and checking the sling is secure prior to lifting off surface. Staff aware to adjust loops to achieve desired position of patient in repositioning sling	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
8.	Demonstrate ability to use Guldmann Repositioning sling to complete supine lateral transfer from bed to other level surface, e.g. stretcher, bed or similar	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
9.	Verbalize proper cleaning and facility laundering protocol for reusable slings, as applicable, and where clean slings are stored.	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
10.	Verbalizes awareness of and how to access resource material available for instruction and guidance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	



Guldmann™

Guldmann Repositioning Sling Skills Check