



Guldmann™

Guldmann Repositioning Sling

Quiz Questions

1. The repositioning sling can be used for transfers from bed to:

- A. Upright Chair (fixed, non-adjustable position)
- B. X-ray table
- C. Stretcher
- D. All Above
- E. B and C

Answer E

2. Which is an appropriate use of a Guldmann Repositioning Sling?

- A. When patient is unable to reposition and roll themselves in bed
- B. When a patient is unable to move self from one lateral surface to another lateral surface
- C. When assisting a patient during recovery from the floor post fall, when the clinical circumstances align
- D. All above

Answer D

3. When using a repositioning sling which of the following are correct?

- A. The specific Repositioning Sling's maximum safe working (SWL) load must not be exceeded
- B. Guldmann Repositioning slings are available in polyester, net, spacer, disposable and Tencel materials
- C. Max weight capacity of is 1100 lbs. unless otherwise stated; e.g. Tencel model and Spacer model
- D. All Above

Answer D

4. The straps on the side of the sling if left hanging too low on the bed, near the floor, are a trip hazard.

- A. T
- B. F

Answer: T

5. When using the sling to turn a patient, which of the following steps are needed:

- A. Use middle 2 straps or all 4 straps on the same side of the bed opposite to the side you want to turn
- B. Side rail up to turning side
- C. Use all straps on both sides
- D. All above
- E. A and B

Answer: E



Guldmann™

6.

When using the sling to boost or reposition patient up in bed, which of the following is correct

- A. Prior to lifting patient, ensure that all straps are secure on hanger bar and patient is supported prior to moving
- B. Ensure all patient equipment, lines and tubes are secure and have sufficient length, prior to lifting or moving
- C. Inform the patient what you are doing
- D. All of the above

Answer D

7. **Which care tasks can be accomplished using the repositioning sling?**

- A. Lift patient off of bed surface to zero an integrated bed scale, in order to take a weight
- B. Lift patient to swap standard bed to a specialty bed model
- C. Lift patient to assist in changing or replacing linen
- D. All of the above

Answer D

8. **Center contrast color stitching is at the top and bottom of the repositioning sling to help identify the center or middle of sling in relationship to the bed as well as the patient.**

- A. T
- B. F

Answer A