



Guldmann™

Instructions Twin Turner

Applying / Placement



- 1 Fold either wide end just at the seam, in the direction of the label.



- 2 Fold either side in toward the middle of the sling.



- 3 Fold the remaining side in toward the middle of the sling, overlapping the first side.



- 4 Place the folded end, label side down, onto bed or other care surface. Place hand, palm down on top of the folded end, to compress bed/surface and slide beneath patient lumbar area.



- 5 Unfold sling, then attach Velcro strap to stable portion of bed frame.



- 6 Attach the free end of sling's loops to the lift's hanger bar and then roll/position patient.