



Guldmann™

Guldmann Twin Turner Sling – Quiz

1. Which care tasks would you select the Guldmann Twin Turner to help you perform:

- A. Assessment of patients skin upon admission and periodically
- B. Changing soiled linen while patient is on bed/surface
- C. Wound dressing changes
- D. All of the above

Answer D

2. Which size of the Twin Turner sling should you use to turn a 500lb. person?

- A. The Large size Twin Turner 225kg (560lbs)
- B. The Bariatric size of the Twin Turner 500kg (1,100lbs)
- C. The size which is appropriate in both SWL (safe working load) and in dimension to the person's body mass distribution

Answer C

3. How do you know when a Guldmann Twin Turner should be taken out of service?

- A. One of the Velcro anchor straps is missing
- B. Chose the incorrect size for patient
- C. The label is missing on the Twin Turner
- D. A and C

Answer D

4. Should the Guldmann Twin Turner be anchored to the bed side rail?

- A. Only for a 80lb patient
- B. When a patient can help turn themselves
- C. If the rail seems strong enough to support the patients weight
- D. None of the above

Answer: D

5. You are using a Twin Turner and it is soiled while cleaning a patient what do you do with the Guldmann Twin Turner when task is complete?

- A. Single patient use slings can be discarded
- B. Reusable/washable Twin Turners are put into the appropriate linen hamper
- C. Always discard all slings after use
- D. Use sling for the next patient
- E. A and B

Answer: E

Guldmann Twin Turner Sling – Quiz

6. When do you explain use for and purpose of the Twin Turner to your patient?

- A. Only when you have time to give an explanation
- B. Only when there are family members in the room
- C. Only for patients that may complain about the use of slings
- D. None of the above

Answer: D

7. What should you do if you need to use the Twin Turner to turn and clean a patient but do not remember how to use it properly?

- A. Ask your strongest co-worker to help manually turn the patient
- B. Ask your unit peer leader or co-worker to review how to apply and use the Twin Turner
- C. Leave cleaning for someone that knows how to use the Twin Turner
- D. Do a quick review of the instructions and video for this sling which are available on the available learning platform or Guldmann Academy
- E. B and D

Answer: E

8. What side of the support surface should the Twin Turner be anchored to for a turn to sidelying?

- A. Both sides
- B. The opposite side of the support surface you are turning the patient towards
- C. The same side of the support surface you are turning the patient towards
- D. If the patient is under 100lbs the Twin Turner doesn't need to be anchored

Answer: C

9. Per manufacturer guidelines what should be inspected for before use?

- A. Broken or worn stitching
- B. Tears or fraying of the straps
- C. Changes in the material consistency
- D. Knots in the straps
- E. All of the above

Answer: E

10. Why is it important to position lower extremity and support the patient's shoulder while turning the patient with the Twin Turner Sling?

- A. The patient will be more comfortable
- B. There will be less strain on the patient's spine
- C. There will be less pressure against the patient's skin
- D. All of the above

Answer: D